
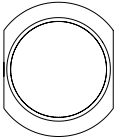

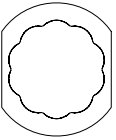






Straumann® Guided Surgery Instruments

Quick Guide

T-sleeves and Drill Handle compatibility

For the surgical planning with Straumann® BLX and Straumann® TLX, two different T-sleeve types are available*:

Conventional T-sleeve		Self-locking T-sleeve*	
			
Stainless steel		PEEK	
In Straumann® coDiagnostiX®, conventional T-sleeves are colored green.		In Straumann® coDiagnostiX® version 10.6 or higher, self-locking T-sleeves are colored grey.	
			
Round cylinder (new) Article number starting with 026.xxxx		Self-locking cylinder Article number starting with 034.xxx	

Ensure that the correct combination of T-sleeve and Drill Handle is selected for implant surgery.






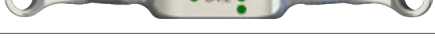



Drill Handles with self-locking cylinders are only compatible with self-locking T-sleeves (PEEK).


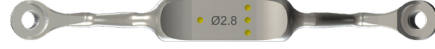
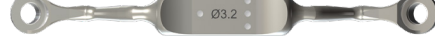

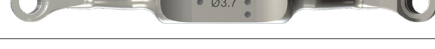


When using conventional T-sleeves (stainless steel), make sure to use the new Drill Handles with the round cylinders.

*In Straumann® coDiagnostiX® version 10.6 and higher, the self-locking T-sleeves (PEEK) are set by default; conventional T-sleeves (stainless steel) can be selected manually as an alternative option.

Article List:

Drill Handles for use with conventional T-sleeves (available April 2022)		
Art. No.	Image	Article
026.0147		Drill Handle, Ø2.2 mm, 1 mm/3 mm
026.0148		Drill Handle, Ø2.8 mm, 1 mm/3 mm
026.0149		Drill Handle, Ø3.2 mm, 1 mm/3 mm
026.0150		Drill Handle, Ø3.5 mm, 1 mm/3 mm
026.0151		Drill Handle, Ø3.7 mm, 1 mm/3 mm
026.0152		Drill Handle, Ø4.2 mm, 1 mm/3 mm
034.053V4		Conventional, height 5 mm, Ø5 mm, Stainless steel, packaging 4 pieces

The new Drill Handles can also be use with the Sleeves Art. No. 034.050V4.

Drill Handles for use with self-locking T-sleeves		
Art. No.	Image	Article
034.291		Drill Handle, Ø2.2 mm, 1 mm/3 mm
034.292		Drill Handle, Ø2.8 mm, 1 mm/3 mm
034.293		Drill Handle, Ø3.2 mm, 1 mm/3 mm
034.294		Drill Handle, Ø3.5 mm, 1 mm/3 mm
034.295		Drill Handle, Ø3.7 mm, 1 mm/3 mm
034.296		Drill Handle, Ø4.2 mm, 1 mm/3 mm
034.299V4		Self-locking, height 5 mm, Ø5 mm, PEEK, packaging 4 pieces

For information on the Modular Cassette and selection of required instruments for all implant lines, please refer to the *Straumann® Modular Cassette Selection Guide (702824/en)*.



International Headquarters

Institut Straumann AG
 Peter Merian-Weg 12
 CH-4002 Basel, Switzerland
 Phone +41 (0)61 965 11 11
 Fax +41 (0)61 965 11 01
www.straumann.com

© Institut Straumann AG, 2022. All rights reserved.

Straumann® and/or other trademarks and logos from Straumann® mentioned herein are the trademarks or registered trademarks of Straumann Holding AG and/or its affiliates.