


Straumann® Guided Adapter, FIBA

Quick Guide

New instruments for fine implant bed preparation.


FIBA compatible Profile Drills and Taps for freehand and guided surgery.

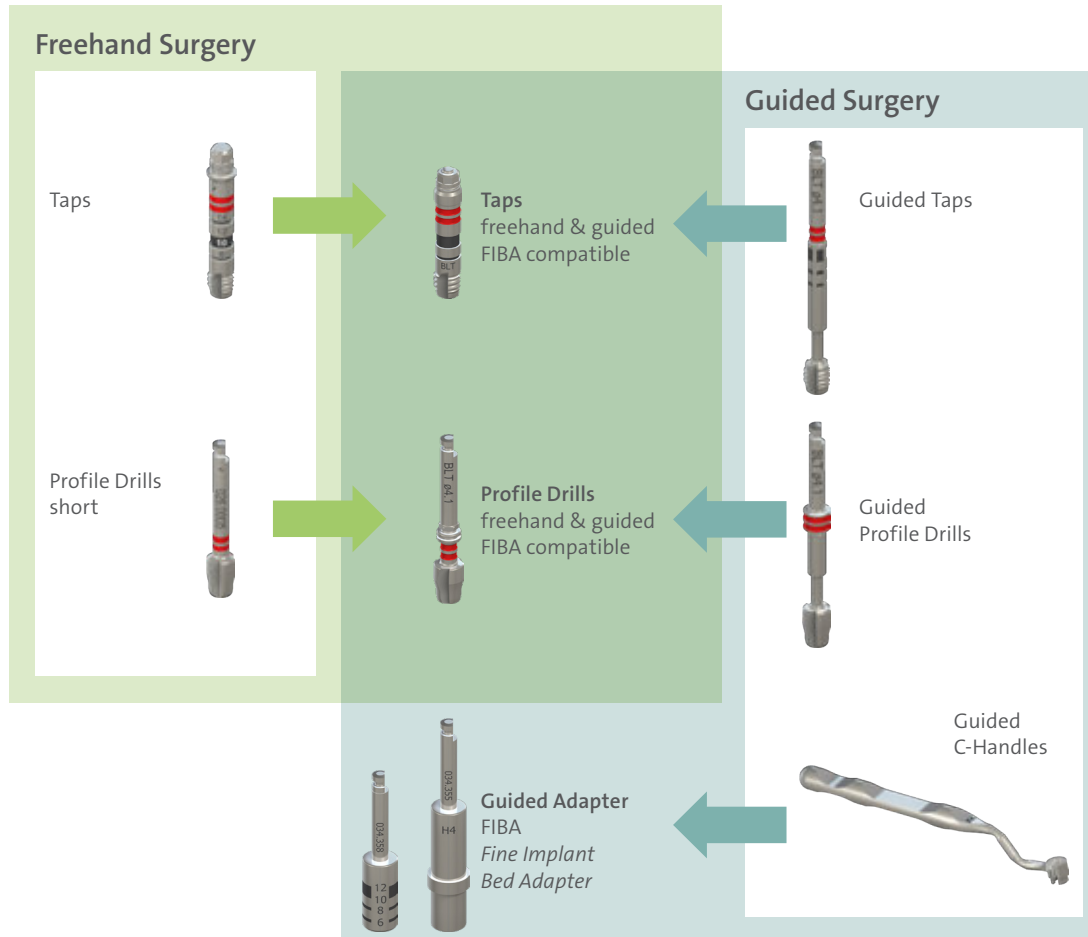

Applicable to





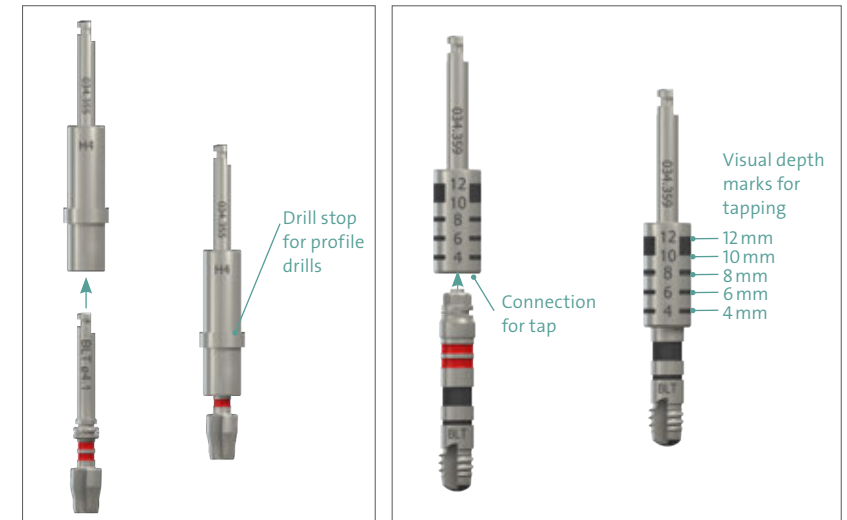


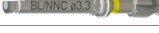














By connecting to the respective Guided Adapter, FIBA, the instruments become guided instruments.



Only Profile Drills and Taps labeled to be "FIBA compatible" are to be used with the Guided Adapter, FIBA. Always use the correct Guided Adapter according to the T-sleeve position (H2, H4, H6). Using the incorrect Guided Adapter or a non-FIBA compatible component may result in an incorrect drilling depth.

Article List:




	Art. No.	Picture	Article
Straumann® Tissue Level Implants 	034.332		SP Profile Drill, RN, short, FIBA compatible, Ø3.3 mm
	034.333		SP Profile Drill, RN, short, FIBA compatible, Ø4.1 mm
	034.351		S/SP Tap, short, FIBA compatible, Ø3.3 mm
	034.352		S/SP Tap, short, FIBA compatible, Ø4.1 mm
	034.353		S/SP Tap, short, FIBA compatible, Ø4.8 mm
Straumann® Bone Level Implants 	034.327		BL/NNC Profile Drill, short, FIBA compatible, Ø3.3 mm
	034.328		BL Profile Drill, short, FIBA compatible, Ø4.1 mm
	034.329		BL Profile Drill, short, FIBA compatible, Ø4.8 mm
	034.348		BL/NNC Tap, short, FIBA compatible, Ø3.3 mm
	034.349		BL Tap, short, FIBA compatible, Ø4.1 mm
	034.350		BL Tap, short, FIBA compatible, Ø4.8 mm
Straumann® BLT Implants 	034.324		BLT Profile Drill, short, FIBA compatible, Ø3.3 mm
	034.325		BLT Profile Drill, short, FIBA compatible, Ø4.1 mm
	034.326		BLT Profile Drill, short, FIBA compatible, Ø4.8 mm
	034.345		BLT Tap, short, FIBA compatible, Ø3.3 mm
	034.346		BLT Tap, short, FIBA compatible, Ø4.1 mm
	034.347		BLT Tap, short, FIBA compatible, Ø4.8 mm




International Headquarters

Institut Straumann AG
 Peter Merian-Weg 12
 CH-4002 Basel, Switzerland
 Phone +41 (0)61 965 11 11
 Fax +41 (0)61 965 11 01
 www.straumann.com

© Institut Straumann AG, 2022. All rights reserved.


Straumann® and/or other trademarks and logos from Straumann® mentioned herein are the trademarks or registered trademarks of Straumann Holding AG and/or its affiliates.

Guided Adapter for Profil Drill, FIBA			
Art. No.	Picture	Article	Sleeve position
034.354		Guided Adapter for Profile Drill, FIBA, H2, handpiece, stainless steel	H2
034.355		Guided Adapter for Profile Drill, FIBA, H4, handpiece, stainless steel	H4
034.356		Guided Adapter for Profile Drill, FIBA, H6, handpiece, stainless steel	H6

Guided Adapter for Tap, FIBA			
Art. No.	Picture	Article	Sleeve position
034.357		Guided Adapter for Tap, FIBA, H2, handpiece, stainless steel	H2
034.358		Guided Adapter for Tap, FIBA, H4, handpiece, stainless steel	H4
034.359		Guided Adapter for Tap, FIBA, H6, handpiece, stainless steel	H6

For manual profile drilling and tapping with a Ratchet use the Connector for Ratchet, art. no. 034.005.

For information on the Modular Cassette and selection of required instruments for all implant lines, please refer to the *Straumann® Modular Cassette Selection Guide* (702824/en).

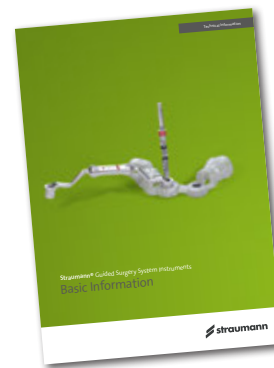

For information on the Straumann® Guided Surgery Instruments, please refer to the *Straumann® Basic Information on Guided Surgery Instruments* (702526/en).