





## 3Shape Trios & ClearCorrect – Anbindung einrichten


# 1. ClearCorrect als Labor anlegen

Falls noch nicht eingerichtet


# 3Shape Communicate – ClearCorrect als Labor einrichten

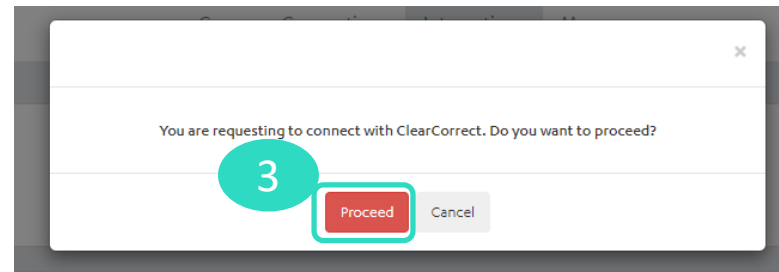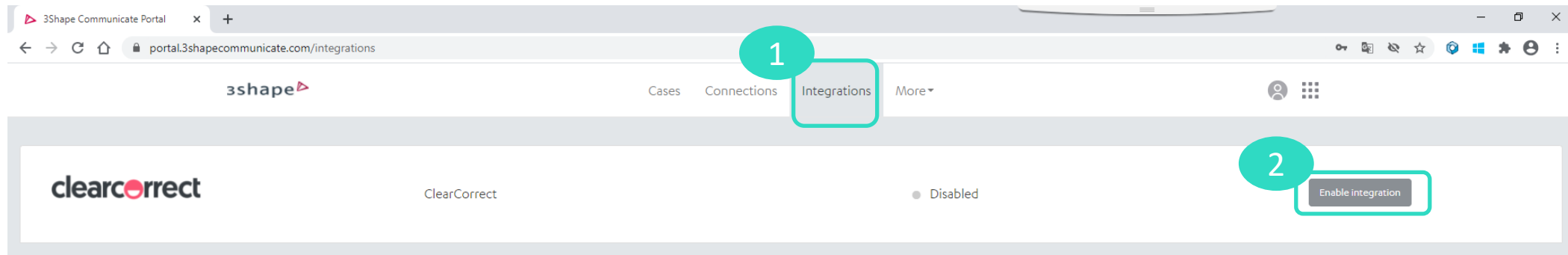

# 3Shape Communicate – ClearCorrect als Labor einrichten

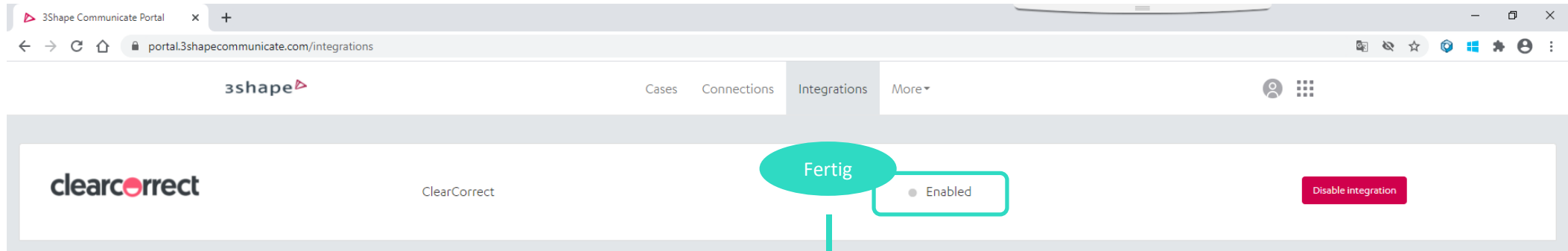

ClearCorrect Kachel kann zukünftig für neue Fälle gewählt werden

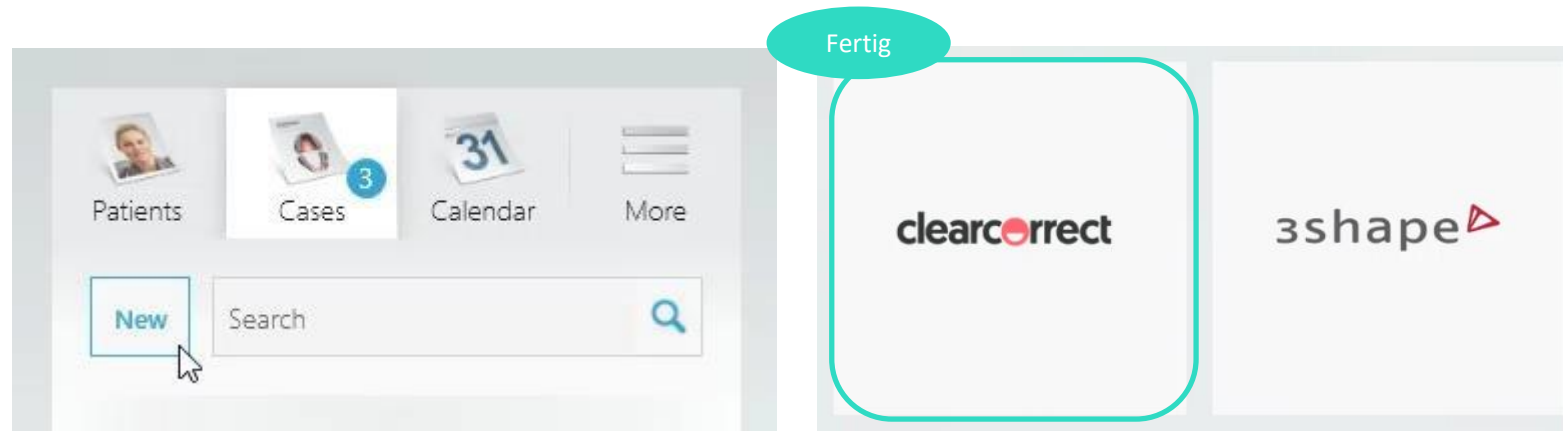


## 2. ClearCorrect Doktorportal mit 3Shape Communicate verbinden

# ClearCorrect Konto mit TRIOS Scanner verbinden


The screenshot displays the ClearCorrect user interface. At the top, a navigation bar includes links for 'RETAINER BESTELLEN' and 'NEUEN FALL ANLEGEN', along with a notification bell and the user's name 'Lea Sonntag'. A dropdown menu for the user name is open, showing 'Mein Konto' and 'Abmelden'. On the left, a sidebar menu lists 'Mein Konto', 'Praxen', 'Passwort', 'Rufnummern', 'Benachrichtigungen', 'PREP', 'TRIOS® Anbindung', and 'Mitarbeiter'. The 'TRIOS® Anbindung' option is highlighted with a red circle and the number '2'. The main content area is divided into two sections. The top section, titled 'Mein Konto', contains a form for 'PERSÖNLICHE DATEN' with fields for 'Vorname \*' (Lea), 'Name \*' (Sonntag), 'Akademischer Grad/Titel \*' (med. dent.), 'Fachgebiet \*' (Anderer Facharzt), and 'Benutzername \*'. The bottom section, titled 'PREP', provides information about the Pre-Release Evaluation Program and includes a checkbox for 'Ich habe die PREP Teilnahme- und Vertraulichkeitsvereinbarung gelesen und stimme zu'. A red button labeled 'Werden Sie Mitglied unseres PREP-Teams' is positioned to the right of the checkbox. Below this, the 'TRIOS® Anbindung' section contains a link labeled 'hier' (highlighted with a red circle and the number '3') for connecting the account to a 3Shape Communicate account.

# ClearCorrect Konto mit TRIOS Scanner verbinden

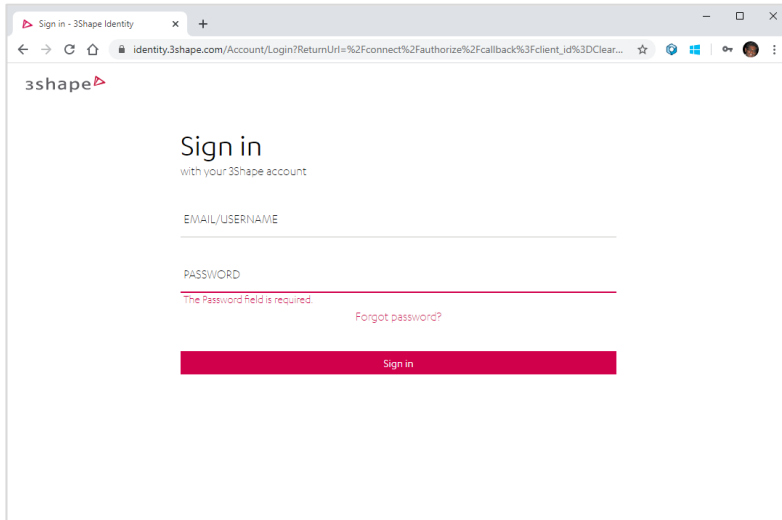

4

3Shape Fenster öffnet sich automatisch.  
3Shape Communicate Nutzernamen und  
Passwort eingeben.

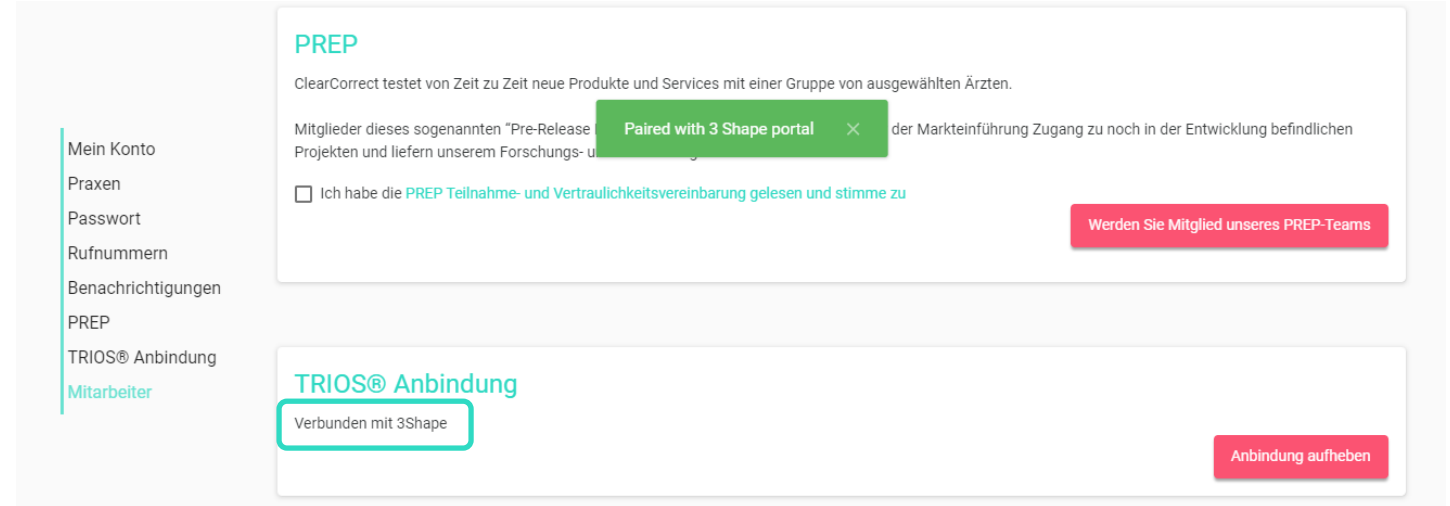

5

Bei erfolgreicher Eingabe, verbindet sich das ClearCorrect Konto mit 3Shape  
Communicate.

### **3. Die Anbindung nutzen und Scans im ClearCorrect Doktorportal hochladen**



# Scans im ClearCorrect Doktorportal hochladen

Patienten scannen und an ClearCorrect Labor schicken: keine ClearCorrect Fallnummer nötig).

Im ClearCorrect Doktorportal kann der Scan dann direkt aus einem Dropdownmenü ausgewählt werden. Bitte beachten Sie das Auswahlmenü beschränkt sich auf die letzten 10 gescannten Patienten.

