




# Install ClearCorrect App & Send Scans



Workflow Screenshot Walkthrough

# Install App and Initiate Pairing

Open Case and Click the ClearCorrect App Icon to Verify API Connection

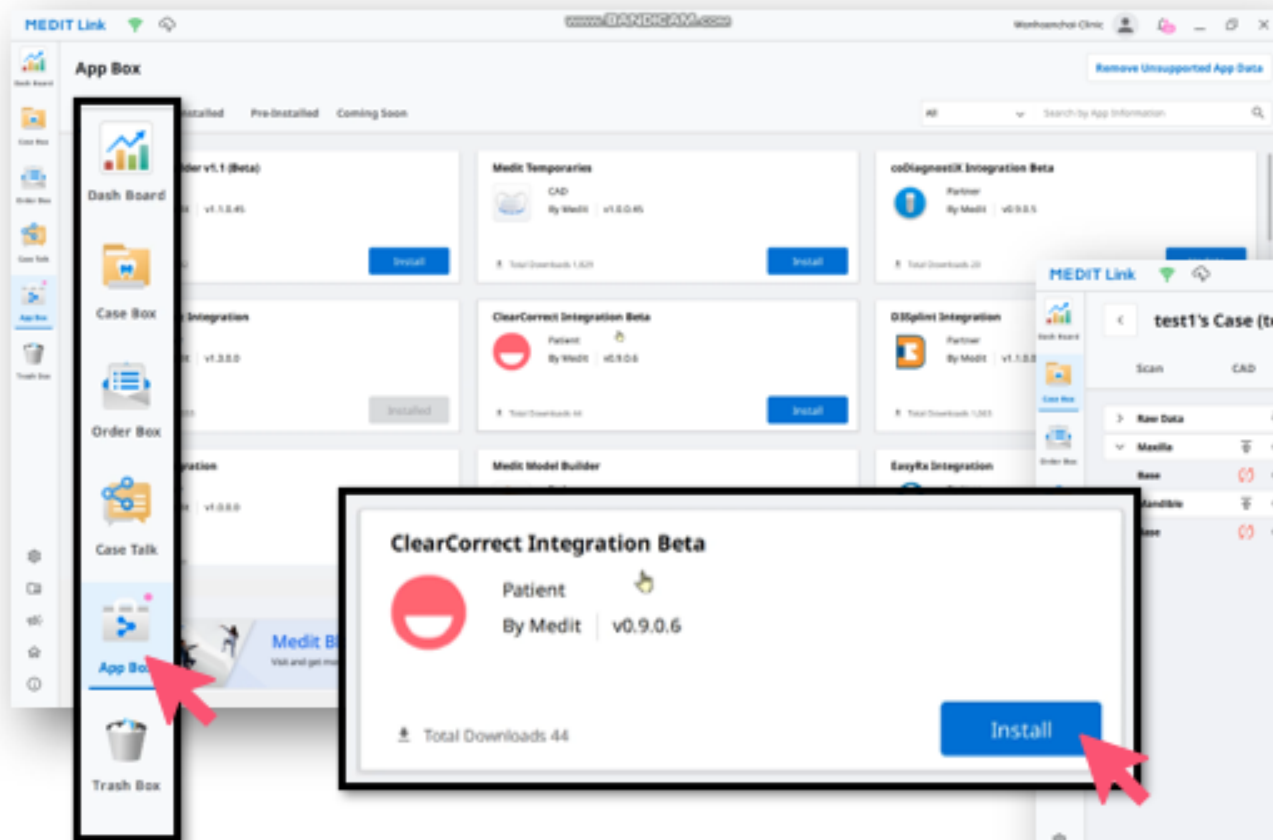

Go to "App Box" & install ClearCorrect Integration

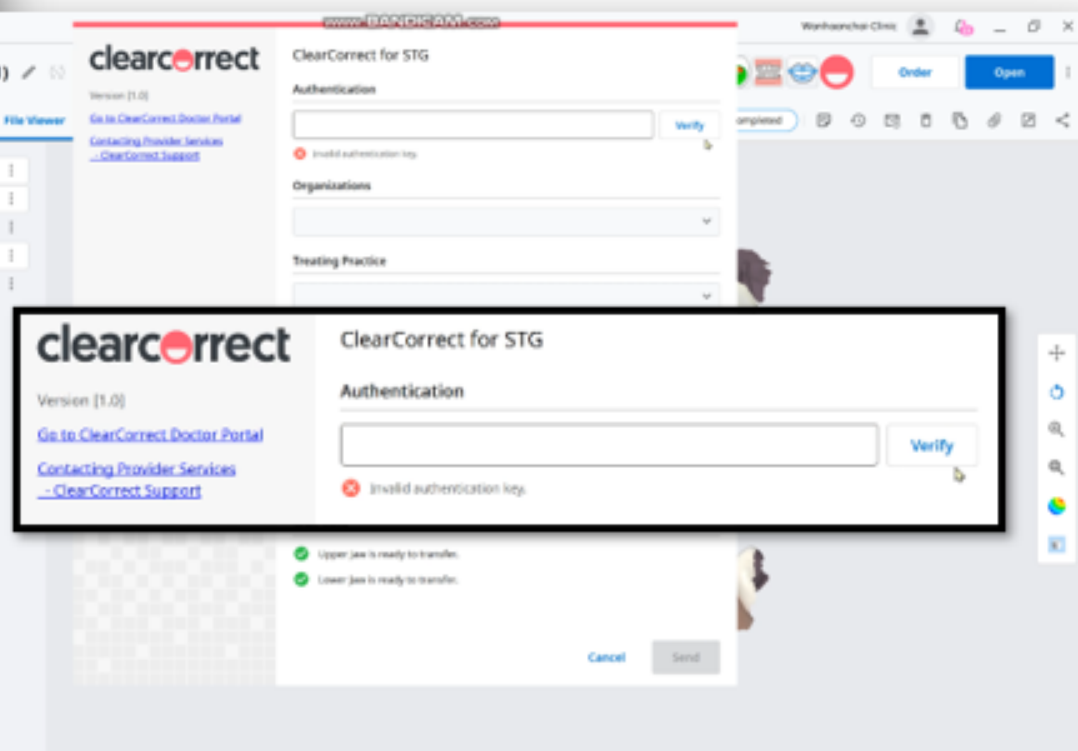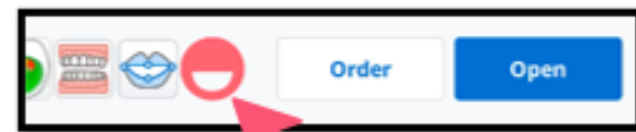

# Generate API Key

Login to Doctor Portal, Open My Account. Navigate to "API Keys" Section. Add new key for MEDIT and copy the "API Key"

The screenshot shows the ClearCorrect Doctor Portal interface. The navigation menu on the left includes: My Account, Practices, Password, Phone Numbers, Email Notifications, PREP, Pairing, Collaborator(s), and API Keys. The 'My Account' dropdown menu is open, showing 'My Account' and 'Logout'. The 'API Keys' section is highlighted, showing a table with columns for Name and API Key. A red arrow points to the '+ Add' button. Another red arrow points to the 'Copy API Key' button. A third red arrow points to the 'API Key' value in the table.

Name	API Key
Medit Test2	617017c6-14c2-4889-a25f-6805455d6380
Medit Test	4f5da602-83c7-49d7-8c7b-1abf751da09d
Test	617017c6-14c2-4889-a25f-6805455d6380

# Verify API Key and Send Scans

The image displays two sequential screenshots of the ClearCorrect web interface. The left screenshot shows the 'Authentication' section with an API key entered in a text field and a red arrow pointing to the 'Verify' button. The right screenshot shows the 'Patient Information' section with fields for 'First Name' and 'Last Name' filled out, and a red arrow pointing to the 'Send' button. A large red arrow points from the 'Verify' button in the first screenshot to the 'Send' button in the second screenshot.

**Step 1:** Paste API Key into Authentication Field and Click "Verify"

**Step 2:** Fill out Patient Information and Click "Send"

# Take Scans

The image displays a user interface for managing orders. On the left, the 'Manage Orders' page features a table with columns for Order #, Name, Last Activity, Status, Category, and Case type. A modal window is overlaid on the table, showing a form for 'InputFirstName' and 'InputLastName' with a 'CONTINUE' button. On the right, the 'Create an order' page shows a multi-step process with a progress indicator and various input fields for patient information and treatment details. Red arrows indicate the flow from the 'Manage Orders' table to the modal, and from the modal to the 'Create an order' form.

Order #	Name	Last Activity	Status	Category	Case type
1000397	InputFirstName InputLastName	21/10/2021	Submission incomplete	Action needed	Recommendation
1000387	Ivan Petrov	20/10/2021	Submission incomplete	Action needed	Recommendation
1000370	FirstName LastName	20/10/2021	Submission incomplete	Action needed	Recommendation
1000367	Ivan Petrov	20/10/2021	Submission incomplete	Action needed	Recommendation

Return to “Manage Orders” page on Dr Portal. Case should be created, select “Continue” and complete prescription workflow to submit case for treatment setup.



Thank you