

How to Send a Case to a Lab and a Patient.

After you have scanned and validated a case, you can now send it to the lab by following the steps below:


Step 1. First select the **Send** icon from the workflow at the top of the screen. A 3D overview of the case will be shown in the center of the screen.

Step 2. You can examine the order in more detail by clicking **View** in the lower left corner to see the scans and patient details

Step 3. To send the case to the patient, press on the **Send to Patient** option.

Note: To be able to use this option, remember to save the patient's email address in the patient profile.

Step 3. Before being sent to the designated lab the scans will be postprocessed.

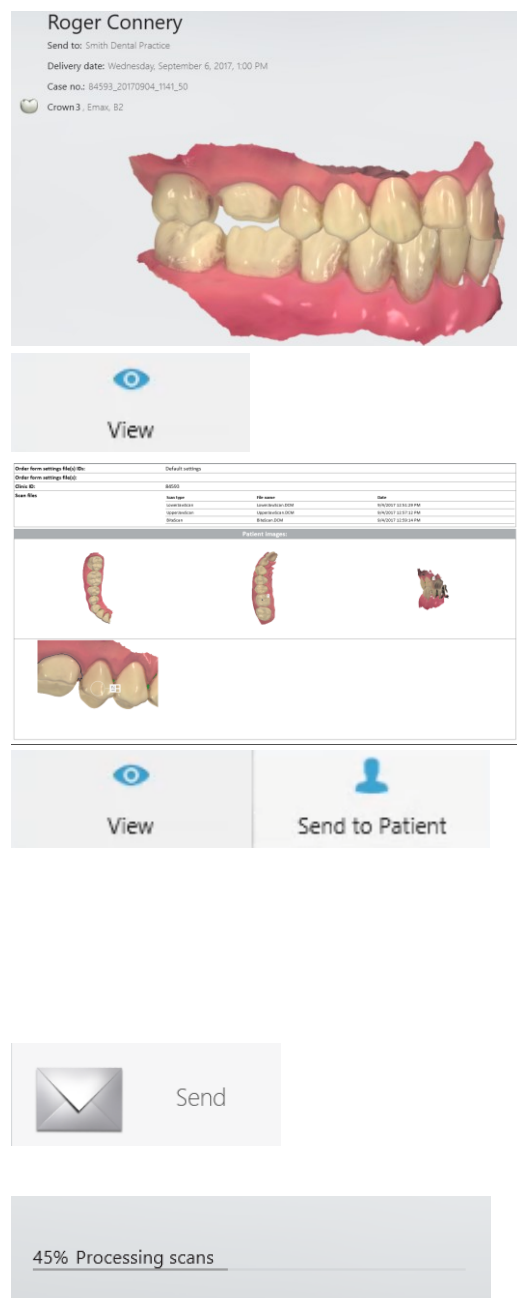

Step 4. When the case has been successfully sent a green tick will appear.



Step 5. You can monitor the status of the case by using 3shape Communicate.