PRO Resins Crown X A1
1000 g REF 019.3220 | 500 g REF 019.3247
PRO Resins Crown X A2
1000 g REF 019.3221 | 500 g REF 019.3248
PRO Resins Crown X A3
1000 g REF 019.3222 | 500 g REF 019.3249
PRO Resins Crown X A3.5
1000 g REF 019.3223 | 500 g REF 019.3250
PRO Resins Crown X B1
1000 g REF 019.3224 | 500 g REF 019.3251
PRO Resins Crown X C1
1000 g REF 019.3225 | 500 g REF 019.3252
PRO Resins Crown X Bleach
1000 g REF 019.3219 | 500 g REF 019.3246

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en INSTRUCTIONS FOR USE

Description:
PRO Resins Crown X is a flowable, light-curing, acrylate-based composite material for the additive production of anterior and posterior restorations. **PRO Resins Crown X** is optimized for wavelengths of 385 nm in layer thicknesses of 100 µm and can be used with 3D printers of following manufacturers:

- Straumann [P-Series (P20, P30, P40, P10+, P20+ cartridge, P20+, P30+, P50+), PRO-Series (PRO20)]
- Asiga [PRO4K UV / MAX UV]

The materials satisfy the performance requirements of DIN EN ISO 10477 type 2/ class 2.

When using **PRO Resins Crown X**, the user must follow all the relevant instructions, including the instructions for use, installation directions, operating instructions, user manuals and other related documents for all listed compatible devices. Strict adherence to all the instructions is crucial for guaranteeing a reliable printed result. Ensure that the equipment items are installed, maintained and calibrated in accordance with the manufacturer's operating instructions. Use only the above-mentioned validated and authorized printers.

Composition:
Preparation consisting of acrylates, fillers (50 %/wt., Ø 0,7 µm), initiators and pigments.

Indication:
Additive production of anterior and posterior restorations, e.g.

- Single-tooth crowns,
- veneers, inlays, onlays.

Contraindications:
Patient and user: Products made from **PRO Resins Crown X** should not be used in patients who are known to be allergic to any of the ingredients. No other patient restrictions apply.

Adverse reactions:
Patient and user: Product can cause allergic reactions.
User: Observe warnings.

Interactions:
There are no known interactions.

Processing instructions:

MINIMUM WALL THICKNESS
➤ Occlusal: 1,5 mm in central fissure
➤ Circumferential: 1mm

CONSTRUCTION PROCESS

➤ **Prepare data set:** The orientation and type of support structures can be found in the system supplier's instructions for use.
➤ **Establish variable process parameters:** The appropriate process parameters are selected according to the specifications of the printer system supplier. Ensure that the correct material-specific set of parameters is selected (product, layer thickness, exposure time).
➤ **Fill with material:** Thoroughly stir material with a spatula and fill the reservoir of the device with the material. Then wait 10 min.
➤ **Start construction process.**

CLEANING AND DRYING

➤ **Option 1:** The component is cleaned by removing the adherent resin using a centrifuge. The recommended rotational speed and time is 1500 rpm for 2 minutes.

Recommendation 1: Laboratory centrifuge with tube holder, e.g.

- SBS-LZ-4000/20-6 from Steinberg
- XC 2450, from Premiere

→ Wrap objects in cellulose cloth, place centrally in the tube and centrifuge, then turn the tube through 180° and centrifuge again.

Recommendation 2: Plate centrifuge e.g.

- LMC 3000 from Biosan with R-2 swing-out rotor.

→ Wrap objects in cellulose cloth, secure centrally on the plate holder of the rotor and centrifuge, then turn the tube through 180° and centrifuge again.

➤ **Option 2:** The component is cleaned under flowing isopropanol for **max. 5 min.** If necessary, carefully clean the objects with a brush and a spray bottle filled with isopropanol.

Dry component: Blow component with compressed air and leave to dry for 30 min at room temperature or in a drying cabinet adjusted to a temperature of approx. 37°C (99°F). For secondary exposure, the components must be dry and cooled to room temperature.

SECONDARY EXPOSURE

Used printer	Curing Device	Post-Curing
Straumann, Asiga	Otoflash	2000 flashes each for top and bottom¹
	P Cure / PROCURE	Selection of the corresponding material parameter set²

¹ Recommendation: Curing under exclusion of oxygen prevents the formation of the inhibition layer.
² Exposure time: 8 min / vacuum 50 mbar / without nitrogen / heater off / power 100 % / wave length 375+415 nm.

➤ Ensure that the unit used is serviced and calibrated in accordance with the manufacturer's instructions in order to preserve its mechanical properties, color and biocompatibility.

FINAL CLEANING

➤ **Final clean component:** The component is cleaned under flowing isopropanol for **max. 2 min.** If necessary, carefully clean the objects with a brush and a spray bottle filled with isopropanol.
➤ **Dry component:** Blow component with compressed air and leave to dry for 30 min. Alternatively, the objects can be stored in a drying cabinet adjusted to a temperature of approx. 37°C (99°F).

REMOVE SUPPORT STRUCTURE AND FINISH

➤ Remove supports and grind remains of the supports to a smooth surface..
➤ Sandblast inner and outer surfaces of the part with 50 µm corundum at 1.5 bar to remove white layer.

➤ **Customization, characterization, polishing:** Restorations made from **PRO Resins Crown X** can be supplemented and customized with light-curing composites and stains. In such cases, the processing instructions of the respective materials used must be observed. Reductions in the vestibular or incisal area should not exceed 0.3 mm. Restorations made from **PRO Resins Crown X** can be polished to a high gloss finish according to the usual dental techniques.

➤ FINISHING COMPLETED

Further processing instructions:
The fit of the printed restorations must be checked and assessed by the dental technician (on the model) and dentist (in the patient's mouth). Before placing it in the patient's mouth, the restoration can be cleaned with alcohol.

Fixation:
Restorations can be fixed with self-adhesive cements (e.g. RelyX™ Unicem 2 / 3M ESPE) or resin cements (e.g. Variolink Esthetic DC / Ivoclar Vivadent) with primers (e.g. Adhese Universal / Ivoclar Vivadent). Observe the processing protocol of the materials used, particularly including the protocol for the pre-treatment of the restoration.

Warnings:
Any deviation from the processing process described here can jeopardize the biocompatibility, functionality and safety of the final product.
Contains skin sensitizing substances (acrylates, phosphine oxide, TPO, Skin Sens. 1B > 0.1%), contains reproductive toxic substances (diboron trioxide, Repr. 1B > 0.1%). Obtain special instructions before use.
Please always observe the current safety data sheet!
Applies to the uncured material: May cause allergic skin reactions. May damage fertility. May damage the unborn child. Avoid contact with uncured material. Wear protective gloves/protective clothing/eye protection. If on skin: Wash with plenty of water and soap.
Do not breathe dust/vapors/spray. Switch on an extraction system during finishing. If exposed or concerned: Get medical advice/attention.
The unpolymerized material is harmful to the environment. Avoid release to the environment. Collect spillage.
Always observe the relevant country-specific health and safety regulations and legal requirements when handling hazardous substances.

Storage instructions / shelf life:

Protect from exposure to light. May polymerize spontaneously. Store between 4°C and 25°C (39.2°F and 77°F). Keep containers tightly closed. Do not use after the expiration date. See imprint/label for expiration date.

Dispose instructions:

Disposal of contents/container in accordance with local regulations.
The product was developed for use in dentistry and must be used according to the instructions for use. Serious incidents must be reported to DeltaMed GmbH and the responsible authority. For the Summary of Safety and Clinical Performance (SSCP) please consult the Eudamed database (https://ec.europa.eu/tools/eudamed) or contact sscp@deltamed.de.

	Batch code		Manufacturer
	Catalogue number		Keep away from sunlight
	Use-by date		Exclamation mark
	Consult instructions for use		Environmental hazard
	Temperature limit		Health hazard
	Medical Device		Stir before use
	Distributor		

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- couronne unitaires,
- facettes, inlays, onlays.

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