

Quick-Guide Implant Removal with new Placement Instrument



1



Open at the perforation and remove blister.

2



Open the blister.
Attention: this removes the sterile barrier.

3



Remove the container and bring it in a upright position.

4



Push the lid sideways to remove it.

5



Fix the implant by pressing lightly on the sides.

6



Insert the new placement instrument without pressure and turn clockwise until the instrument slides into the connection.

Video Microcone



Video Quattrocone



MEDENTiKA[®]
A Straumann Group Brand

Quick-Guide Implant Removal with new Placement Instrument



You hear or feel a click, the implant is secured on the placement instrument.



Release the pressure and remove the secured implant.



Insert the implant into the implant bed.




Turn the container upside down, where the closure screw is placed.

Not included with QUATTROCONE30



Unscrew the closure screw.

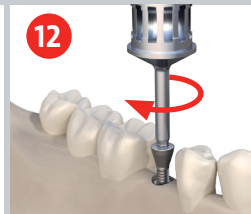

Place the closure screw in the implant.