

—
CLINICAL
CASE

Grand Morse

Full arch immediate fixed reconstruction.

Patient's Medical History

Patient is female, aged 58, and has a history with migraines and antidepressants.

Planning

Full arch.
Immediate Loading Protocol.
With Flap Access Technique.



©neodent

Description of the procedure

1. Full upper arch clearance
2. Placement of tilted implants to avoid sinus augmentation.
3. Placement of multiunit mini conical abutments
4. Placement of impression copings (open tray)
5. Floss and pattern resin to secure a rigid matrix
6. Multifunctional appliance impression.
7. Placement of sutures.
8. Placement of healing caps.

Prosthetic Description:

Acrylic on a metal strengthener - Temporary prosthesis
Acrylic on PEEK - Final Prosthesis.

Result description and/or conclusion

In conclusion, the patient was extremely conscious of her upper teeth and smile; she wanted a "Hollywood" appearance smile.

The patient therefore chose a bleached tooth option for her final restoration and she wanted her new smile to be dramatically different from her original teeth. This patient was highly motivated and was delighted with the end result, which has given her a lot more self confidence.



PROF. JOE BHAT BDS FDS RCS

MClinDent MRD RCSEd

drjoebhat@gmail.com

UNITED KINGDOM



Hon. Visiting Professor,
Specialist Prosthodontist and Oral
Surgeon, Director at Moor Park
Specialist Dental Centre, ITI Fellow.

Other doctors that participated in the
procedure:

DR CHANDNI PATTNI


www.neodent.com.br/grandmorse

Full arch immediate fixed reconstruction.

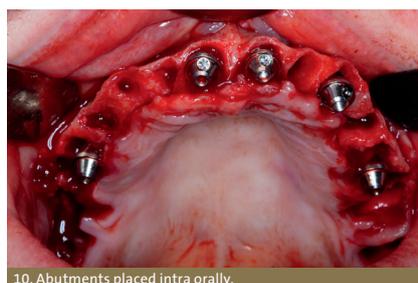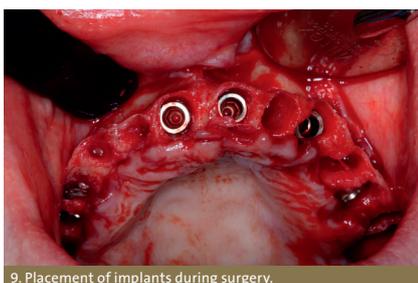

Neodent Grand Morse

Full arch immediate fixed reconstruction.

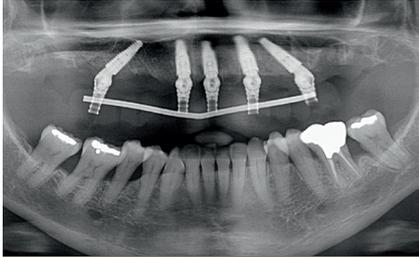

15. 1 week post surgery.



16. 1 week post surgey.



17. 1 week post surgery.



18. 1 week post surgery.



19. Final hybrid bridge.



20. Bleached teeth shade chosen at patients request for much lighter teeth.



21. Bleached teeth shade chosen at patients request for much lighter teeth.