


PROSTHETIC TORQUE VALUES

Product	Torque
Closure Screw	Hand Tight
Healing Cap (TL)	Hand Tight
Healing Abutment (BL & BLX)	Hand Tight
Healing Abutment Customisable BL	Hand Tight
Temporary Abutment (PEEK TL & BL)	15-35 Ncm
Temporary Abutment (Titanium TL, BL & BLX)	15-35 Ncm
Abutments – Final (TL, BL & BLX)	35 Ncm
Occlusal Screws (TL, BL & BLX)	15 Ncm

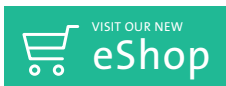

Australia



New Zealand

bit.ly/StraumannShopAU or bit.ly/StraumannShopNZ

FREE Shipping on all eShop orders. Excludes express and cold freight.

Straumann Pty Ltd/Straumann New Zealand Limited

93 Cook Street, Port Melbourne VIC 3207, Australia

AU Toll Free 1800 660 330 **NZ Toll Free** 0800 408 370

Email marketing.au@straumann.com

www.straumann.com.au | www.straumann.co.nz





EDUCATION AND ENABLEMENT



It is our mission to be the benchmark for professional training and continuing development in implant dentistry. We and our partners offer a comprehensive education program in all disciplines of implant dentistry, and with the latest technology available. This covers all aspects from tooth preservation to implant-borne restorations, as well as oral tissue regeneration – for all specialisations, on all skill levels.

**DISCOVER OUR
EDUCATION PROGRAM**



For more than 30 years, Straumann has enjoyed a close and fruitful relationship with its independent scientific partner, the International Team for Implantology (ITI).

Join the ITI Global Community
iti.org/community/become-a-member

