



Straumann® Dental Implant System
Basic Information



Straumann is the industrial partner of the ITI (International Team for Implantology) in the areas of research, development, and education.

Contents

1. The Straumann® Dental Implant System	3	5. Healing Phase	52
1.1 Overview	3	5.1 Healing phase duration	52
1.2 Implant types	4	6. Additional Information on Instruments	53
1.2.1 Straumann Standard Implant	4	6.1 Surgical instruments	53
1.2.2 Straumann Standard Plus Implant	4	6.1.1 Depth marks on Straumann instruments	53
1.2.2.1 Straumann® Standard Plus Narrow Neck CrossFit® Implant	4	6.1.2 Single-patient pilot drills and twist drills	54
1.2.2.2 Straumann® Standard Plus Short Implant	4	6.1.3 Straumann drill stop – precise depth control	54
1.2.3 Straumann Tapered Effect Implant	4	6.1.4 Straumann surgical cassette	55
1.2.4 Straumann Bone Level Implant	4	6.1.5 Ratchet	56
1.3 Implant-abutment connections	5	6.1.6 Holding key	57
1.3.1 Straumann synOcta® Morse taper connection	5	6.1.7 SCS screwdrivers	57
1.3.2 Straumann Narrow Neck CrossFit® connection	5	6.2 Osteotomes	58
1.3.3 Straumann Bone Level CrossFit connection	6	6.3 Cleaning and Care of Instruments	60
1.3.4 The Loxim® Transfer Piece	7	7. Appendix	61
2. Indications and Contraindications	8	7.1 Related documentation	61
2.1 List of abbreviations	8	7.2 Important guidelines	62
2.2 Implant specific indications	9	8. Index	63
2.2.1 Roxolid® Implants	9		
3. Preoperative Planning	11		
3.1 Implant position	11		
3.1.1 Mesiodistal implant position	12		
3.1.2 Orofacial implant position	16		
3.1.3 Coronoapical implant position	17		
3.2 Planning aids	19		
3.2.1 Mesiodistal and orofacial space requirements	19		
3.2.2 Determining the vertical bone availability	21		
3.2.3 Surgical drill template	22		
4. Surgical Procedures	26		
4.1 Implant bed preparation	26		
4.1.1 Basic implant bed preparation	27		
4.1.2 Final implant bed preparation	31		
4.1.3 Examples for fine implant bed preparation			
Straumann Standard and Standard Plus Implants	34		
4.2 Opening the implant package	38		
4.3 Placing the implant	40		
4.3.1 Additional information for implants with the Loxim Transfer Piece	45		
4.4 Soft tissue management	46		
4.4.1 Submucosal healing	46		
4.4.2 Transmucosal healing	48		

About this guide

Basic Information on the Surgical Procedures for the Straumann® Dental Implant System provides dental practitioners and related specialists with the essential steps regarding surgical treatment, planning, and procedure.

The manual is divided into the following main parts:

- The Straumann Dental Implant System
- Indications and Contraindications
- Preoperative Planning
- Surgical Procedures
- Additional Information on Instruments
- Appendix
- Index

For information on the Straumann Guided Implants and the Straumann Guided Surgery system, please refer to the following brochure: *Basic Information on Straumann Guided Surgery*, NAMLIT 1006.

For further information regarding the Straumann Dental Implant System, visit our comprehensive website at: www.straumann.us

1. The Straumann® Dental Implant System.

1.1 OVERVIEW

The Straumann Dental Implant System offers seven implant types with diverse body and neck designs, ranging from the classic Soft Tissue Level Implant to the Bone Level Implant and some additional hybrid solutions. All implants can be placed with the instruments from the Straumann Surgical Cassette while using very similar surgical procedures.

Straumann® Dental Implant System						
Tissue Level Implants					Bone Level Implants	
S	SP	NNC	TE	SPS	BL	BLT
Straumann® Standard Implant (S)	Straumann® Standard Plus Implant (SP)	Straumann® Standard Plus Narrow Neck CrossFit® Implant (NNC)	Straumann® Tapered Effect Implant (TE)	Straumann® Standard Plus Short Implant (SPS)	Straumann® Bone Level Implant (BL)	Straumann® Bone Level Tapered Implant (BLT)*

Straumann dental implants are available in four endosteal diameters: Ø 2.9 mm, Ø 3.3 mm, Ø 4.1 mm, and Ø 4.8 mm. A unified color code simplifies identification of instruments and implants.

Color coding		
	blue	Endosteal implant diameter 2.9 mm
	yellow	Endosteal implant diameter 3.3 mm
	red	Endosteal implant diameter 4.1 mm
	green	Endosteal implant diameter 4.8 mm

1.2 IMPLANT TYPES

1.2.1 STRAUMANN® STANDARD IMPLANT – THE CLASSIC SOFT TISSUE LEVEL IMPLANT

Straumann Standard Implants have a smooth neck section of 2.8 mm and are especially suitable for classic single-stage procedures, where the implant is placed at soft tissue level and not covered with soft tissue during the healing phase. The Standard Implant uses the Straumann synOcta® connection together with its corresponding prosthetic components, the Straumann synOcta portfolio and the Straumann Solid Abutment. The thread pitch on the Standard Implants measures 1 mm for the Ø 3.3 mm implants, and 1.25 mm for all other diameters.

1.2.2 STRAUMANN STANDARD PLUS IMPLANT – THE IMPLANT FOR FLEXIBLE PLACEMENT

Straumann Standard Plus Implants have a shorter smooth neck section of 1.8 mm that allows flexible coronal implant placement in combination with trans- or subgingival healing. This offers the dental surgeon additional options that are particularly useful in the anterior tooth region of the maxilla, where esthetic demands are high. Similar to Straumann Standard Implants, this implant type uses the Straumann synOcta connection together with its corresponding prosthetic components, the Straumann synOcta portfolio and the Straumann Solid Abutment. The thread pitch on the Standard Plus Implants measures 1 mm for the Ø3.3 mm implants, and 1.25 mm for all other diameters.

1.2.2.1 STRAUMANN STANDARD PLUS NARROW NECK CROSSFIT® IMPLANT

The Narrow Neck CrossFit® (NNC) Implant is a 3.3 mm diameter implant with a narrow prosthetic platform. Its internal connection provides expanded prosthetic options and solutions for treatment in the upper and lower jaw, wherever space is limited. The NNC Implant is a Standard Plus (SP) Soft Tissue Level Implant with a machined neck of 1.8 mm in height. It combines the innovations of the Roxolid® material and the SLActive® surface technology, thus giving you more confidence when placing small diameter-implants. The implant body and thread design is the same as the Straumann 3.3 mm Bone Level NC Implant. The NNC is available from 8 mm to 14 mm and in Roxolid material. Narrow Neck CrossFit Implants use the Narrow Neck CrossFit (NNC) prosthetic components.

1.2.2.2 STRAUMANN® STANDARD PLUS SHORT IMPLANT

The Straumann® Standard Plus Short Implant is Straumann's shortest implant. The implant features a Standard Plus design for easy oral hygiene in the posterior regions, synOcta® internal connection compatibility with the existing Tissue Level prosthetic portfolio, Bone Level thread to increase the implant-to-bone contact. The most advanced Straumann technology combined with ease of use.

1.2.3 STRAUMANN TAPERED EFFECT IMPLANT – THE IMPLANT FOR IMMEDIATE PLACEMENT

Straumann Tapered Effect Implants have a special anatomical design, which combines a cylindrical shape in its apical region and a conical shape in the coronal region, making this implant particularly suitable for immediate or early implantation following extraction or loss of natural teeth. With the smooth neck section of 1.8 mm, healing can occur trans- or subgingivally. Tapered Effect Implants have a Straumann synOcta connection. Hence, the prosthetic components of the Straumann synOcta portfolio and the Straumann Solid Abutment can be used. The thread pitch of 0.8 mm provides excellent primary stability.

1.2.4 STRAUMANN BONE LEVEL IMPLANT – STRAUMANN EXPERTISE APPLIED AT BONE LEVEL

Straumann Bone Level Implants are suitable for bone level treatments in combination with trans- or subgingival healing. The implant's rough surface extends to the top of the implant and the connection is shifted inwards. The Bone Level Implant uses a conical-cylindrical connection, the CrossFit connection, together with its corresponding prosthetic CrossFit components from the Bone Level product portfolio. A cylindrical outer contour and a thread pitch of 0.8 mm, that tapers off in the coronal part of the implant, provide excellent primary stability.

1.2.5 STRAUMANN BONE LEVEL TAPERED IMPLANT

The Straumann Bone Level Tapered Implant features the established and clinically proven Straumann® Bone Control Design® and the CrossFit® connection together with its corresponding prosthetic CrossFit® components from the Bone Level Implant product portfolio. It has a tapered apex with three cutting notches, which makes this implant particularly suitable for situations involving soft bone or fresh extraction sockets where primary stability is key.

1.3 IMPLANT-ABUTMENT CONNECTIONS

1.3.1 STRAUMANN® SYNOCTA® MORSE TAPER CONNECTION


The Straumann synOcta concept was introduced worldwide in 1999, using the well-known Morse taper design principle developed in 1986. The mechanically locking friction fit of the Straumann synOcta internal connection, with an 8° cone and an octagon for the repositioning of prosthetic parts, shows improved performance over traditional external connections.

The Straumann synOcta connection is available for all Straumann Standard, Standard Plus, and Tapered Effect Implants with the Regular Neck (RN) and Wide Neck (WN) platform.



1.3.2 STRAUMANN NARROW NECK CROSSFIT® CONNECTION

The Narrow Neck CrossFit (NNC) Implant is a 3.3 mm diameter implant with a narrow prosthetic platform. The NNC Implant is a Standard Plus (SP) Soft Tissue Level Implant with a machined neck of 1.8 mm in height. The implant body and thread design is the same as the Straumann 3.3 mm Bone Level NC Implant.

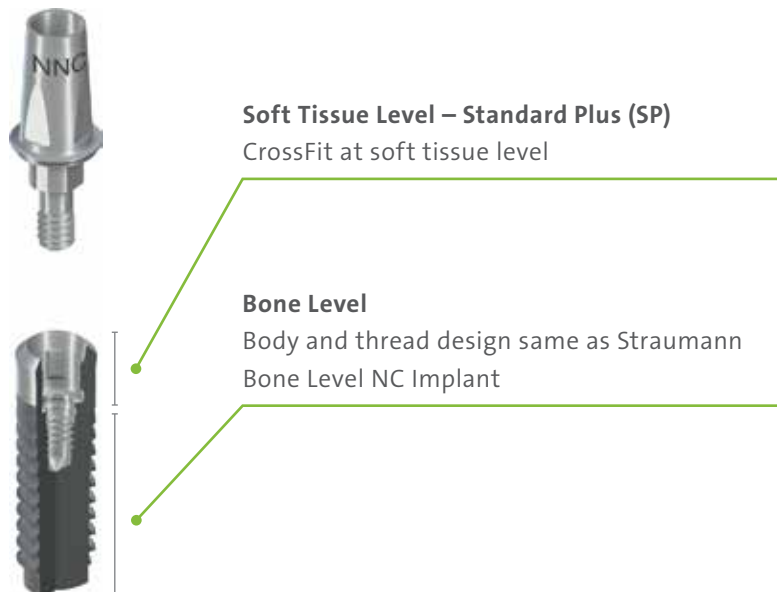

1.3.3 STRAUMANN® BONE LEVEL CROSSFIT® CONNECTION

The CrossFit connection of Straumann Bone Level Implants applies the knowledge and benefits from the Straumann synOcta® Morse taper connection to the connection requirements at bone level. Similar to the Straumann sy Octa connection, the mechanically locking friction fit of the 15° conical-cylindrical CrossFit connection with four internal grooves has excellent long-term stability under all loading conditions. The CrossFit connection is available for Straumann Bone Level Implants only.

Straumann Bone Level Ø4.1 mm and Ø4.8 mm Implants have the same connection, the regular CrossFit connection (RC), and share the same secondary components. Straumann Bone Level Ø3.3 mm Implants feature the narrow CrossFit connection (NC). Straumann Bone Level Ø2.9 mm implants feature the Small Crossfit connection (SC). The corresponding secondary components are color-coded:

- Blue = SC connection
- Yellow = NC connection
- Magenta = RC connection



Connection types

NNC: Narrow Neck CrossFit Ø3.5 mm	Ø3.5 mm
RN: Regular Neck Ø4.8 mm	Ø4.8 mm
WN: Wide Neck Ø6.5 mm	Ø6.5 mm
NC: Narrow CrossFit Ø3.3 mm	Ø3.3 mm
SC: Small CrossFit Ø2.9 mm	Ø2.9 mm
RC: Regular CrossFit Ø4.1 and Ø4.8 mm	Ø4.1 mm Ø4.8 mm

1.3.4 EASY HANDLING: THE LOXIM® TRANSFER PIECE

Pre-mounted Loxim Transfer Piece for ease of use

- Secures transport into mouth
- Operates with existing Adapters

Self-retaining

- Detaches with Adapter after implant insertion

Small diameter/short

- Easier access to narrow interdental spaces and the posterior region

Clockwise and counterclockwise turns

- Integrated extraction function in case of implant removal (only during implant insertion)

Alignment Pin

- Can be re-inserted into the implant
- Alignment in multiple implant situations

Restoration-safe torque stop

- Designed for ease of implant restoration



2. Indications and Contraindications

To obtain more information about indications and contraindications related to each implant, please refer to the corresponding instructions for use. Instructions for use can also be found on ifu.straumann.com

2.1 LIST OF ABBREVIATIONS











List of abbreviations
SCS = Screw Carrying System
HDD = Horizontal Defect Dimension
NNC = Narrow Neck CrossFit® connection (3.5 mm)
RN = Regular Neck (4.8 mm)
WN = Wide Neck (6.5 mm)
SC = Small CrossFit connection (for Bone Level Tapered Implants)
NC = Narrow CrossFit connection (for Bone Level Implants)
RC = Regular CrossFit connection (for Bone Level Implants)
S = Standard
SP = Standard Plus
SPS = Standard Plus Short
TE = Tapered Effect
BL = Bone Level
BLT = Bone Level Tapered

2.2 IMPLANT SPECIFIC INDICATIONS

2.2.1 ROXOLID® IMPLANTS

Straumann offers Roxolid material in all lengths and diameters, an alloy composed of titanium and zirconium, which offers high strengths. The Roxolid material was designed to meet the needs of dental implantologists, and is available with the SLActive® and SLA® surfaces.

Specific indications for Straumann® Roxolid® Implants

Implant type		Distinctive features	Minimal ridge width*	Minimal gap width**	Available lengths
BLT Ø2.9 mm SC Roxolid SLActive		<ul style="list-style-type: none"> Small diameter implant for upper lateral incisors and all lower incisors 	5 mm	5 mm	10–14 mm
SP Ø3.3 mm NNC SLActive/SLA†		<ul style="list-style-type: none"> Small-diameter implant for narrow interdental spaces and bone ridges 	5.5 mm	5.5 mm	8–14 mm
S Ø3.3 mm RN SLActive/SLA†		<ul style="list-style-type: none"> Ideal in cases with restricted ridge width 	5.5 mm	7 mm	8–16 mm
SP Ø3.3 mm RN SLActive/SLA†					8–14 mm
TE Ø3.3 mm RN SLActive/SLA†		<ul style="list-style-type: none"> For oral endosteal implant indications in the maxilla and mandible, for functional and esthetic rehabilitation of edentulous and partially edentulous patients Alternative in dental gaps where the roots of adjacent teeth are close together, where implants with a greater endosteal diameter are contraindicated 	7 mm	7 mm	8–14 mm
BL Ø3.3 mm NC SLActive/SLA†		<ul style="list-style-type: none"> Small-diameter implant for narrow interdental spaces and ridges 	5.5 mm	5.5 mm	8–14 mm
BLT Ø3.3 mm NC Roxolid SLActive/ SLA†		<ul style="list-style-type: none"> Small diameter implant for narrow interdental spaces and bone ridges 	5.5 mm	5.5 mm	8–18 mm
S Ø4.1 mm RN SLActive/SLA		<ul style="list-style-type: none"> For oral endosteal implant indications in the maxilla and mandible, for functional and esthetic rehabilitation of edentulous and partially edentulous patients 	6 mm	7 mm	6–16 mm
SP Ø4.1 mm RN SLActive/SLA					6–14 mm
SP Ø4.1 mm RN SLActive®***		<ul style="list-style-type: none"> Open-end situations in the mandible with moderate to severely atrophic bone resorption (always splinted, one implant per unit) 	6 mm	7 mm	4 mm













* Minimal ridge width: Minimal orofacial ridge width, rounded off to 0.5 mm.

** Minimal gap width: Minimal mesial-distal gap width for a single-tooth restoration, between adjacent teeth, rounded off to 0.5 mm.

*** Only available in Roxolid®

† Small-diameter implants are not recommended for the posterior region.

Specific indications for Straumann® Roxolid® Implants

Implant type		Distinctive features	Minimal ridge width*	Minimal gap width**	Available lengths
TE Ø4.1 mm RN SLActive/SLA		<ul style="list-style-type: none"> For oral endosteal implant indications in the maxilla and mandible, for functional and esthetic rehabilitation of edentulous and partially edentulous patients 	7 mm	7 mm	8–14 mm
BL Ø4.1 mm RC SLActive/SLA		<ul style="list-style-type: none"> For oral endosteal implant indications in the maxilla and mandible, for functional and esthetic rehabilitation of edentulous and partially edentulous patients 	6 mm	6 mm	8–14 mm
BLT Ø4.1 mm NC Roxolid® SLActive®/SLA®		<ul style="list-style-type: none"> For oral endosteal implant indications in the maxilla and mandible, for functional and esthetic rehabilitation of edentulous and partially edentulous patients 	6 mm	6 mm	8–18 mm
SP Ø 4.8 mm RN SLActive®***		<ul style="list-style-type: none"> Open-end situations in the mandible with moderate to atrophic bone resorption (always splinted, one implant per unit) 	7 mm	7 mm	4 mm
S Ø4.8 mm RN SLActive/SLA		<ul style="list-style-type: none"> For oral endosteal implant indications in the maxilla and mandible, for functional and esthetic rehabilitation of edentulous and partially edentulous patients The S/SP Ø4.8 mm Implants are especially suited for wider interdental spaces and ridges 	7 mm	7 mm	6–14 mm
SP Ø4.8 mm RN SLActive/SLA					
S Ø4.8 mm WN SLActive/SLA		<ul style="list-style-type: none"> For oral endosteal implant indications in the maxilla and mandible, for functional and esthetic rehabilitation of edentulous and partially edentulous patients The S/SP Ø4.8 mm Implants are especially suited for wider interdental spaces and ridges S/SP Implants with a WN platform are designed for the reconstruction of teeth with a greater neck diameter 	7 mm	8.5 mm	6–12 mm
SP Ø4.8 mm WN SLActive/SLA					
SP Ø 4.8 mm WN SLActive®/SLA®***		<ul style="list-style-type: none"> Open-end situations in the mandible with severely atrophic bone resorption (always splinted, one implant per unit) 	7 mm	8.5 mm	4 mm
TE Ø4.8 mm WN SLActive/SLA		<ul style="list-style-type: none"> For oral endosteal implant indications in the maxilla and mandible, for functional and esthetic rehabilitation of edentulous and partially edentulous patients TE Ø4.8 mm Implants are especially suited for wider interdental spaces and ridges 	8.5 mm	8.5 mm	10–14 mm
BL Ø4.8 mm RC SLActive/SLA		<ul style="list-style-type: none"> For oral endosteal implant indications in the maxilla and mandible, for functional and esthetic rehabilitation of edentulous and partially edentulous patients BL Ø4.8 mm Implants are especially suited for wider interdental spaces and ridges 	7 mm	7 mm	8–14 mm
BLT Ø4.8 mm NC Roxolid® SLActive®/SLA®		<ul style="list-style-type: none"> For oral endosteal implant indications in the maxilla and mandible, for functional and esthetic rehabilitation of edentulous and partially edentulous patients BLT Ø4.8 mm Implants are especially suited for wider interdental spaces and ridges 	7 mm	7 mm	8–18 mm

* Minimal ridge width: Minimal orofacial ridge width, rounded off to 0.5 mm.

** Minimal gap width: Minimal mesial-distal gap width for a single-tooth restoration, between adjacent teeth, rounded off to 0.5 mm.

*** Only available in Roxolid®

3. Preoperative planning

3.1 IMPLANT POSITION

The implant is the focal point of the dental restoration. It provides the basis for planning the surgical procedure. Close communication between the patient, dentist, surgeon and dental technician is imperative for achieving the desired prosthetic result.

To establish the topographical situation, the axial orientation, and the choice of implants, we recommend the following:

- Make a wax-up/set-up on the previously prepared study cast
- Define the type of superstructure

The wax-up/set-up can later be used as the basis for a custom-made X-ray or drill template and for a temporary restoration.

Note

The implant abutments should always be loaded axially. Ideally, the long axis of the implant is aligned with the cusps of the opposing tooth. Extreme cusp formation should be avoided. It can lead to unphysiological loading.

The implant diameter, implant type, position and number of implants should be selected individually, taking the anatomy and spatial circumstances (e.g. malpositioned or inclined teeth) into account. The measurements given here should be regarded as minimum guidelines. Only when the minimum distances are observed is it possible to design the restoration so that the necessary oral hygiene measures can be carried out.

The final hard and soft tissue response is influenced by the position between the implant and the proposed restoration. Therefore, it should be based on the position of the implant-abutment connection. The implant position can be viewed in three dimensions:

- Mesiodistal
- Orofacial
- Coronoapical


3.1.1 MESIODISTAL IMPLANT POSITION

The mesiodistal bone availability is an important factor for choosing the implant type and diameter as well as the interimplant distances in the case of multiple implants. The point of reference on the implant for measuring mesiodistal distances is always the shoulder, being the most voluminous part of the implant. Note that all distances given in this chapter are rounded off to 0.5 mm. The following basic rules must be applied:

Rule 1

Distance to adjacent tooth at bone level:


A minimal distance of **1.5 mm from the implant shoulder to the adjacent tooth** at bone level (mesial and distal) is required.

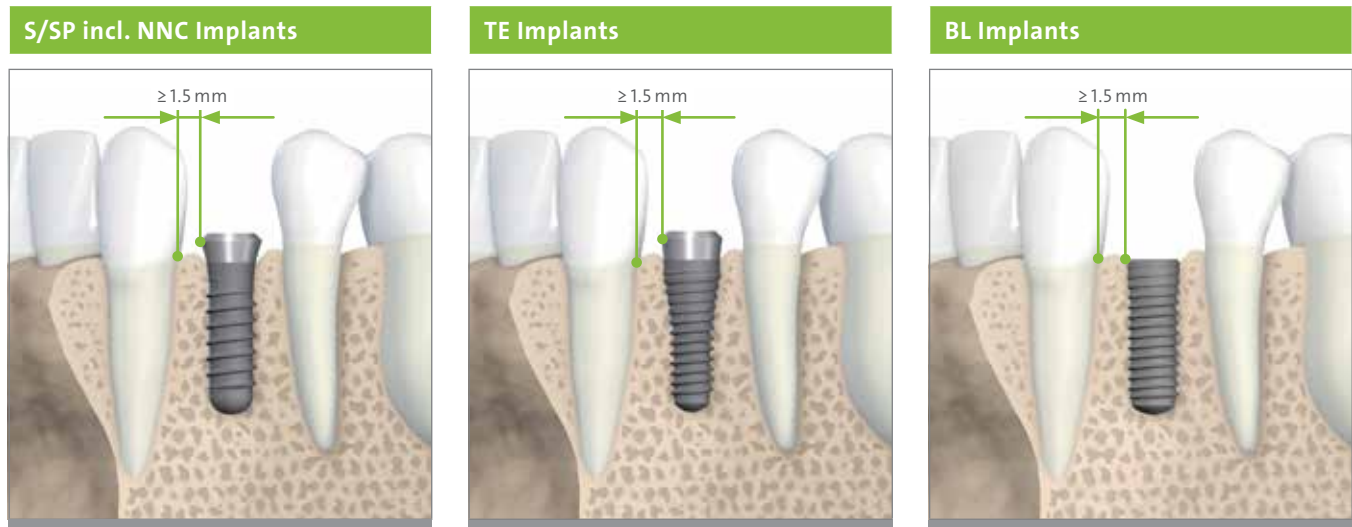

Rule 2

Distance to adjacent implants at bone level:

A minimal distance of **3 mm between two adjacent implant shoulders** (mesiodistal) is required.

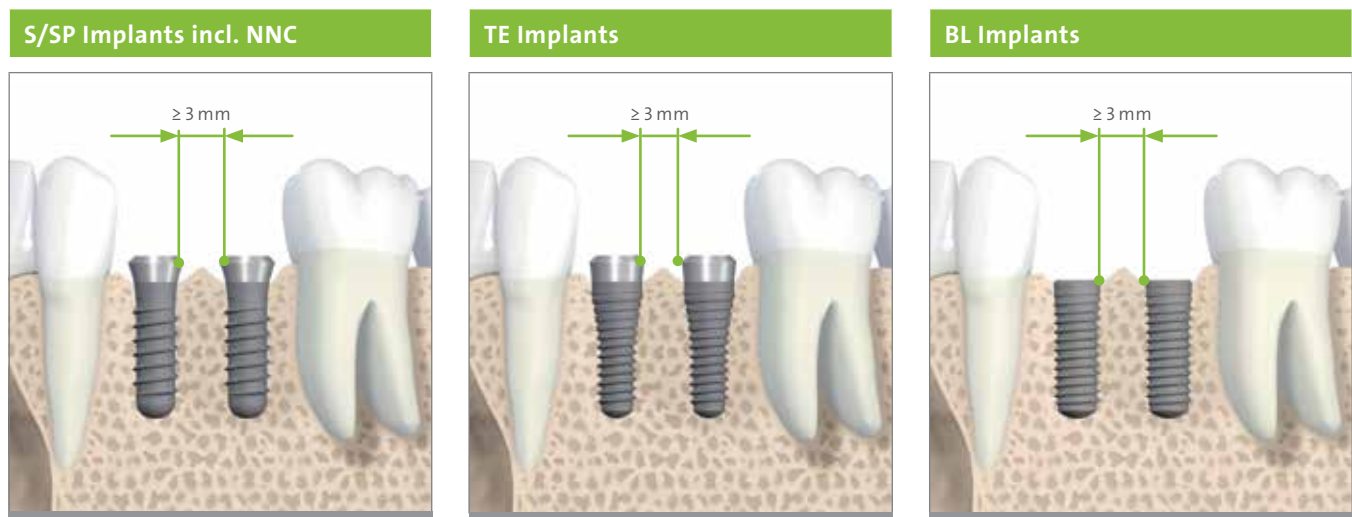


3.1.1.1 EXAMPLES FOR SINGLE TOOTH GAPS

For single tooth restoration, the implant is placed centered within the single tooth gap. The following examples show how Rule 1 is implemented.


Straumann® Standard and Tapered Effect Implants

For Straumann Soft Tissue Level Implants, the gap size has to be considered for the selection of the shoulder diameter (NNC, RN, WN). In order to make use of the gap width in conjunction with Rule 1, the following approximation can be used.

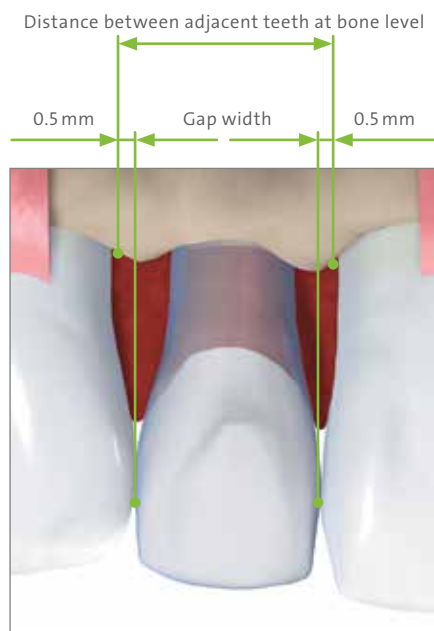

The distance between adjacent teeth at bone level is approximately 1 mm (2×0.5 mm) more than the gap width. Hence, applying Rule 1, the gap width must be 2 mm wider than the implant shoulder.

S/SP/TE Implants

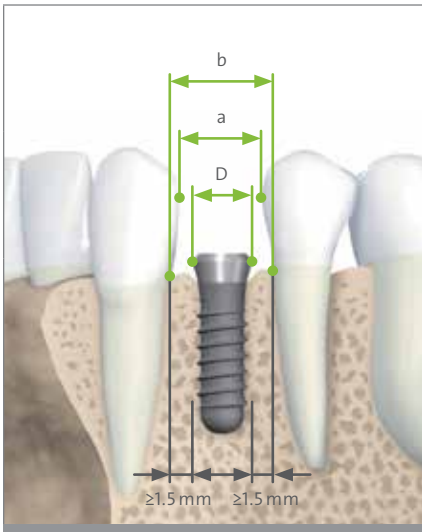

Shoulder diameter D (mm)	Gap width a_{\min} (mm)	Distance between adjacent teeth at bone level b_{\min} (mm)
Ø3.5 (NNC)	5.5	6.5
Ø4.8 (RN)	7.0	8.0
Ø6.5 (WN)	8.5	9.5
Rule	D + 2 mm	D + 3 mm*


* Rule 1 applied on both implant sides

The Diagnostic T (see page 20), applied in the patient's mouth or on the cast, can be used to obtain an initial measurement of the gap width for the choice of the implant shoulder diameter and prosthetic reconstruction.

Straumann® Bone Level implants

For Straumann Bone Level implants, the distance between adjacent teeth at bone level determines the implant diameter.

BL Implants

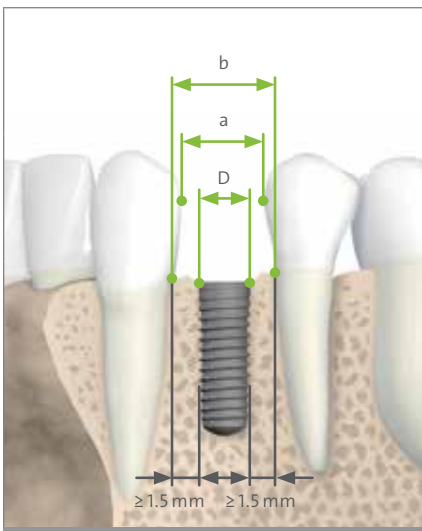


Shoulder diameter D (mm)	Gap width a_{\min} (mm)	Distance between adjacent teeth at bone level b_{\min} (mm)
BL Ø3.3	5.5	6.5
BL Ø4.1	6.0	7.0
BL Ø4.8	7.0	8.0
Rule	D + 2 mm	D + 3 mm*

* Rule 1 applied on both implant sides

3.1.1.2 EXAMPLES OF MULTIPLE TOOTH GAPS


The following examples show how Rules 1 and 2 are implemented in multiple tooth gaps. The measurement is made at bone level from the adjacent tooth to the center of the implant and between implant centers. The minimal distance of 3 mm between two adjacent implant shoulders (Rules 2) is important to facilitate flap adaptation, avoid proximity of secondary components and provide adequate space for maintenance and home-care.

Straumann® Standard, Standard Plus, and Tapered Effect Implants

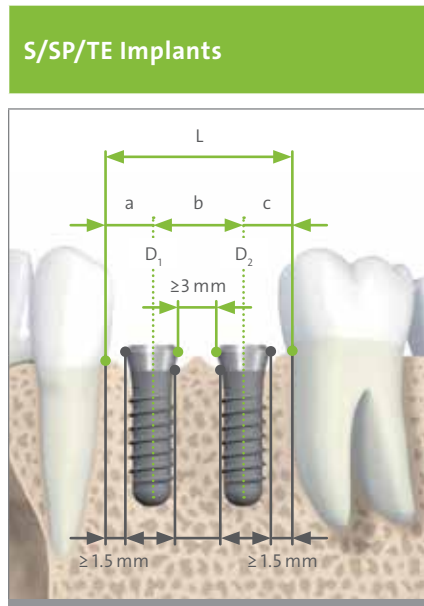

Shoulder diameter D ₁ (mm)	Shoulder diameter D ₂ (mm)	a _{min} (mm)	b _{min} (mm)	c _{min} (mm)	L _{min} (mm)
Ø3.5 (NNC)	Ø3.5 (NNC)	3.0	6.5	3.0	12.5
Ø3.5 (NNC)	Ø4.8 (RN)	3.0	7.0	4.0	14.0
Ø3.5 (NNC)	Ø6.5 (WN)	3.0	8.0	5.0	16.0
Ø4.8 (RN)	Ø4.8 (RN)	4.0	8.0	4.0	16.0
Ø4.8 (RN)	Ø6.5 (WN)	4.0	8.5	5.0	17.5
Ø6.5 (WN)	Ø6.5 (WN)	5.0	9.5	5.0	19.5

Straumann Bone Level Implants

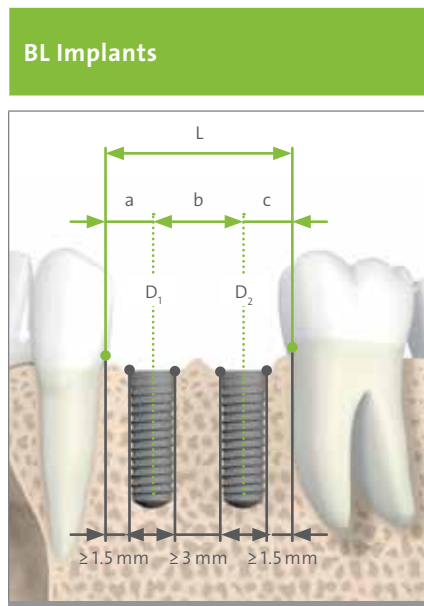



Shoulder diameter D ₁ (mm)	Shoulder diameter D ₂ (mm)	a _{min} (mm)	b _{min} (mm)	c _{min} (mm)	L _{min} (mm)
BL Ø3.3	BL Ø3.3	3.0	6.5	3.0	12.5
BL Ø3.3	BL Ø4.1	3.0	7.0	3.5	13.5
BL Ø3.3	BL Ø4.8	3.0	7.0	4.0	14.0
BL Ø4.1	BL Ø4.1	3.5	7.0	3.5	14.0
BL Ø4.1	BL Ø4.8	3.5	7.5	4.0	15.0
BL Ø4.8	BL Ø4.8	4.0	7.5	4.0	15.5

3.1.2 OROFACIAL IMPLANT POSITION

The facial and palatal bone layer must be at least 1 mm thick in order to ensure stable hard and soft tissue conditions. The minimal orofacial ridge widths for individual implant types are given in the indication tables on pages 9–11. Within this limitation, a restoration-driven orofacial implant position and axis should be chosen such that screw-retained restorations are possible.


Caution

An augmentation procedure is indicated where the orofacial bone wall is less than 1 mm or a layer of bone is missing on one or more sides. This technique should be employed only by dentists who have adequate experience in the use of augmentation procedures.

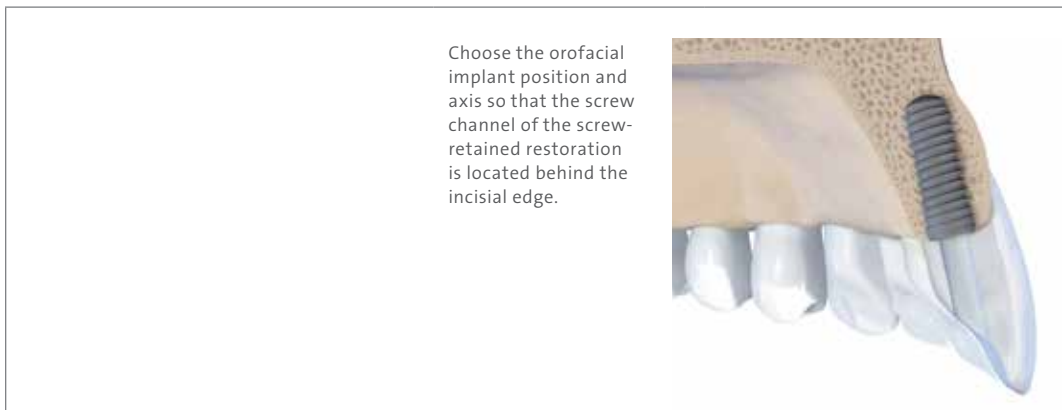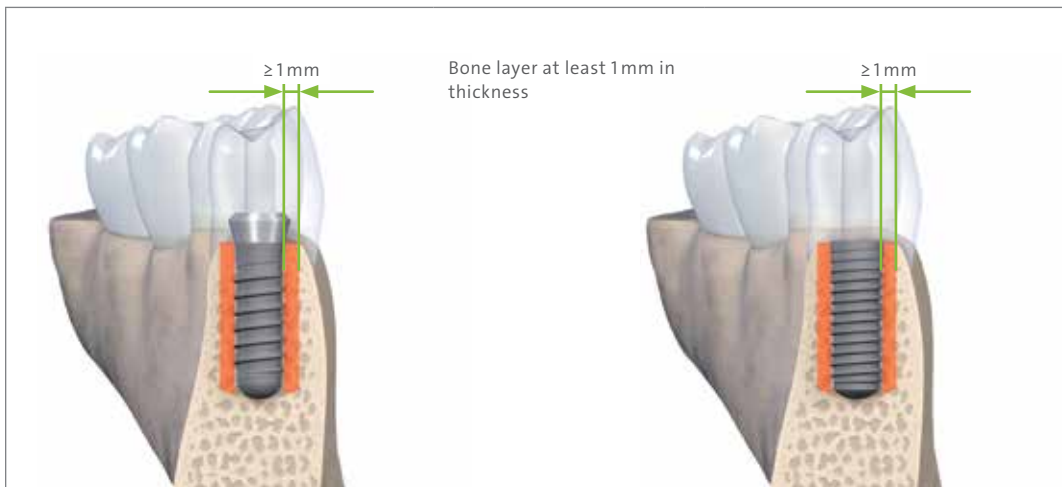

3.1.3 CORONOAPICAL IMPLANT POSITIONING

Straumann® dental implants allow for flexible coronoapical implant positioning, depending on individual anatomy, implant site, the type of restoration planned, and preference. In the anterior area, a deeper coronoapical implant position is better for esthetic reasons. In this situation, the use of Straumann Standard Plus, Tapered Effect or Bone Level Implants is recommended. The following illustration shows the coronoapical implant position for these implants.

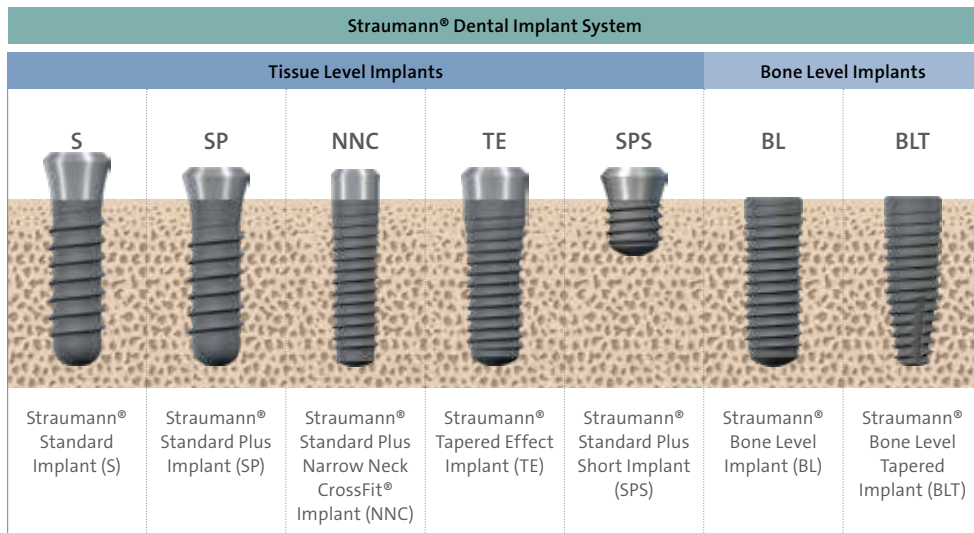

Straumann Standard Implants

Straumann Standard Implants with a smooth neck section of 2.8 mm are submerged in the bone as far as the margin of the SLA®/SLActive® surface.

Straumann Standard Plus and Tapered Effect Implants

Straumann Standard Plus and Tapered Effect Implants with a smooth neck section of 1.8 mm are submerged in the bone as far as the margin of the Straumann SLA/SLActive surface. Optionally they can be placed slightly deeper if necessary.

Ideally, in the esthetic region, the implant shoulder should be positioned about 1 mm apical to the cemento-enamel junction (CEJ) of the contralateral tooth or 2 mm subgingival of the prospective gingival margin (see also references on page 19).


Caution

If a Straumann Standard Plus or a Tapered Effect Implant is inserted deeper than the margin of the Straumann SLA/SLActive surface, the preparation depth must be increased accordingly (see also page 53).

Straumann® Bone Level Implants

Straumann Bone Level Implants are best set with the outer rim of the small 45° sloping edge (chamfer) at bone level.

Ideally, in the esthetic region, the implant shoulder should be positioned about 3–4 mm subgingival of the prospective gingival margin (see also use of Bone Level transfer part on pages 42/44).

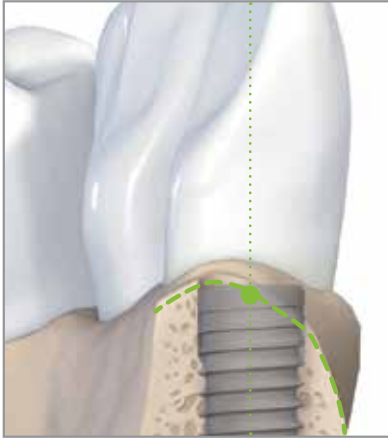

In a scalloped situation, place the mesial/distal point of the outer rim of the implant to bone level. The lingual/palatinal wall will then extend slightly over the top line of the implant. The buccal wall is located somewhat below the implant edge.

For further information regarding surgical procedures in cases pertaining to esthetics, please refer to the following scientific publications:



ITI Consensus Paper

Buser D, Martin W, Belser UC. Optimizing esthetics for implant restorations in the anterior maxilla: anatomic and surgical considerations. Int J Oral Maxillofac Implants. 2004;19 Suppl:43-61.



ITI Treatment Guide

Buser D, Martin W, Belser U. Surgical considerations for single-tooth replacements in the esthetic zone : standard procedure in sites without bone deficiencies. Quintessence Publishing Co. Ltd, Berlin; 2007. 26–37. (ITI Treatment Guide. Implant Therapy in the Esthetic Zone. Single-Tooth Replacements; Vol. 1).

3.2 PLANNING AIDS


3.2.1 MESIODISTAL AND OROFACIAL SPACE REQUIREMENTS

3.2.1.1 DIAGNOSTIC T FOR STRAUMANN® STANDARD, STANDARD PLUS, AND TAPERED EFFECT IMPLANTS

By using the Diagnostic T in the patient's mouth or on the cast, an initial impression of the spatial relations for the choice of the implant shoulder diameter and prosthetic reconstruction can be obtained. The pictograms on the instruments show which arm is used for which measurement. The use of additional planning methods, such as the use of a drill template (see page 23), is recommended.

Note

Currently, a Diagnostic T for Straumann Bone Level Implants is not available.

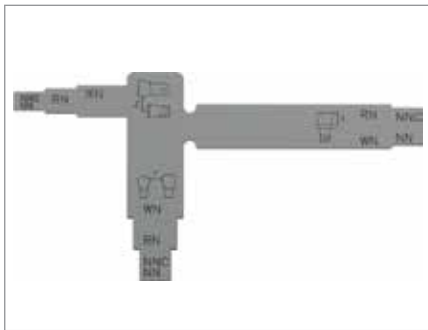


X = Minimum occlusal space requirement
(for the lowest prosthetic restoration option)
Y = Interproximal distance (gap width)
Z = Implant center to adjacent tooth
(½ the gap width)

Implant shoulders:

NNC = Narrow Neck CrossFit® (Ø3.5 mm)
RN = Regular Neck (Ø4.8 mm)
WN = Wide Neck (Ø6.5 mm)



Determining the implant shoulder diameter in a single tooth gap

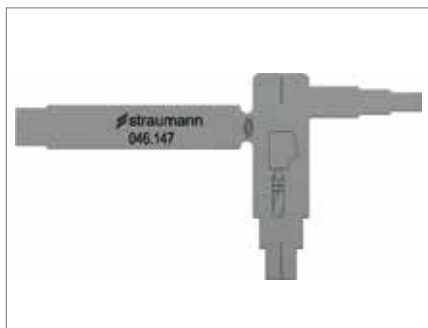

Minimum vertical space requirement for access with surgical instruments



Determining the minimal distance between implant axis and adjacent teeth

3.2.1.2 STRAUMANN® IMPLANT DISTANCE INDICATOR

Two types of Implant Distance Indicators are available:

- For Straumann Standard, Standard Plus and Tapered Effect Implants (Art. No. 046.148)
- For Straumann Bone Level Implants (Art. No. 026.0901)

The four discs of the Implant Distance Indicators display the shoulder diameters of Straumann implants. The Implant Distance Indicators can be used to check the available space before the start of treatment or intraoperatively to mark the desired implant site.



Intraoperative use of the Implant Distance Indicator before flap opening



Round Bur Ø1.4 mm Needle Drill, Ø1.6 mm

After flap opening and precise positioning of the disc(s) at the planned implantation site, it is possible to drill through the perforation in the disc(s) with the Round Bur Ø1.4 mm (Art. No. 044.022) or the Ø1.6 mm Needle Drill (Art. No. 026.0054) in order to mark the center of the implant bed.

Implant Distance Indicator for Straumann Standard, Standard Plus, and Tapered Effect Implants



Straumann Implant Distance Indicator for Straumann Standard, Standard Plus and Tapered Effect Implants (Art. No. 046.148)

	Leg label	Disk diameter	Corresponding implants
Leg 1	RN Ø4.8	Ø4.8 mm	all Regular Neck (RN) Implants
Leg 2	RN Ø4.8	Ø4.8 mm	all Regular Neck (RN) Implants
Leg 3	NNC Ø3.5	Ø3.5 mm	all Narrow Neck CrossFit® (NNC) Implants
Leg 4	WN Ø6.5	Ø6.5 mm	all Wide Neck (WN) Implants

Implant Distance Indicator for Straumann® Bone Level Implants



Straumann Implant Distance Indicator for Straumann Bone Level Implants (Art. No. 026.0901)

	Leg label	Disk diameter	Corresponding implants
Leg 1	BL Ø4.1	Ø4.1 mm	Bone Level Implants Ø4.1 mm
Leg 2	BL Ø4.1	Ø4.1 mm	Bone Level Implants Ø4.1 mm
Leg 3	BL Ø3.3	Ø3.3 mm	Bone Level Implants Ø3.3 mm
Leg 4	BL Ø4.8	Ø4.8 mm	Bone Level Implants Ø4.8 mm

3.2.2 DETERMINING THE VERTICAL BONE AVAILABILITY

The vertical bone availability determines the maximal allowable length of the implant that can be placed. To make it easier in determining the vertical bone availability, the use of an X-ray Template with X-ray Reference Spheres is recommended.



3.2.2.1 X-RAY REFERENCE SPHERE

The X-ray Reference Sphere (Art. No. 049.076V4) has a diameter of 5 mm. The image of the sphere on the X-ray provides the reference value for the magnification scale. To prepare a reference sphere carrying template, the selected implant positions are marked on the study cast. The X-ray Reference Spheres are fixed at the marked points. The vacuum-formed template is then made with the spheres. The subsequent X-ray shows the vertical bone availability and mucosal thickness, from which the corresponding implant length and type can be derived, in consideration of the enlargement factor.

⚠ Warning

Adhere to production requirements of the holding template and ensure that the X-ray Reference Sphere is securely fixed within the holding template.

3.2.3 SURGICAL DRILL TEMPLATE

A custom-made drill template facilitates planning and preparation of the implant bed and enables precise use of the cutting instruments. The planning basis for fabricating this template should be the desired prosthetic result.

3.2.3.1 VACUUM-FORMED DRILL TEMPLATE

A conventional surgical drill template can be produced with the vacuum-formed template components.



The 10 mm-long metal pin functions as the X-ray Reference Pin. After the X-ray Reference Pin is integrated into the template, the planned implant axis and position become visible on the X-ray



The Drill Sleeve is then secured in a drill template

3.2.3.2 X-RAY TEMPLATES

The X-ray Templates are used for measurement and comparison. They also assist the user in selecting the suitable implant type, diameter and length. The following X-ray Templates are available:

- For Straumann® Standard and Standard Plus Implants (Art. No. 150.215)
- For Straumann Tapered Effect Implants (Art. No. 150.230)
- For Straumann Bone Level Implants (Art No. 150.216)
- For Straumann Bone Level Tapered Implants (Art No. 025.0003)

Similar to the distortions that occur in X-rays, the implant dimensions are shown on the individual templates with the corresponding distortion factors (1:1 to 1.7:1).

Determining each magnification factor or scale is facilitated by showing the X-ray Reference Sphere on the template (next to the scale reference).


The first stage consists of comparing the size of the X-ray Reference Sphere on the patient's X-ray with the size of the reference sphere on the template. By superimposing the two pictures, the correct scale can be found. Then, the spatial relations around the implant position are determined and the implant length and insertion depth are established.

⚠ Warning


Use only the X-ray Template specific to the implant type.




Example: Scale 1.1:1 = reference sphere Ø5.5 mm

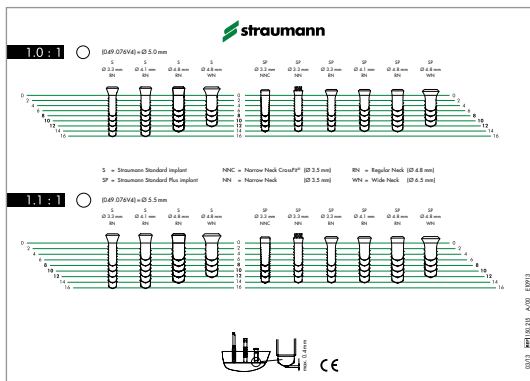


X-ray Template for Straumann Standard and Standard Plus Implants (Art. No. 150.215)

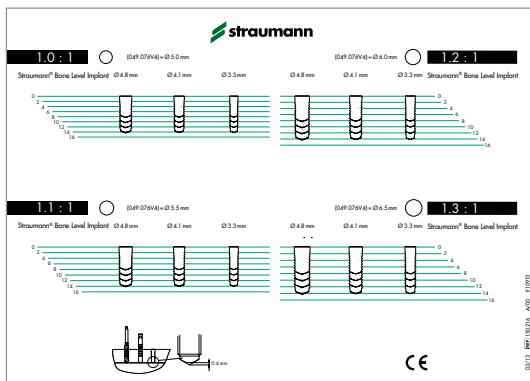

X-ray Template for Straumann Tapered Effect Implants (Art. No. 150.230)

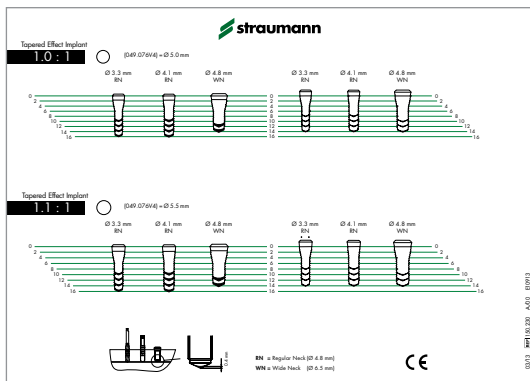

X-ray Template for Straumann Bone Level Implants (Art. No. 150.216)

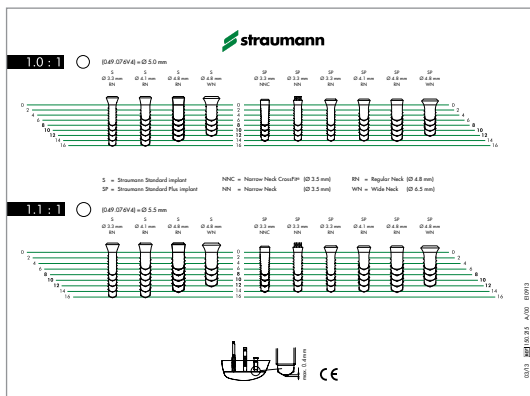

X-ray Template for Straumann Bone Level Tapered Implants (Art. No. 025.0003)

To calculate the effective bone availability the following formula should be used:


$$\frac{\text{X-ray Reference Sphere 5 mm} \times \text{bone availability (X-ray*)}}{\text{Reference sphere diameter on the X-ray}} = \text{effective bone availability}$$

* Taking into consideration all implant-related anatomic structures (e.g. mandibular canal, sinus maxillaris, etc.)

Example for a measured bone availability and reference sphere diameter on the X-ray of 13 mm and 6 mm (+ 20 % distortion), respectively.

$$\frac{5 \text{ mm} \times 13 \text{ mm}}{6 \text{ mm}} = 10.8 \text{ mm}$$

Additional length of the drill tip:

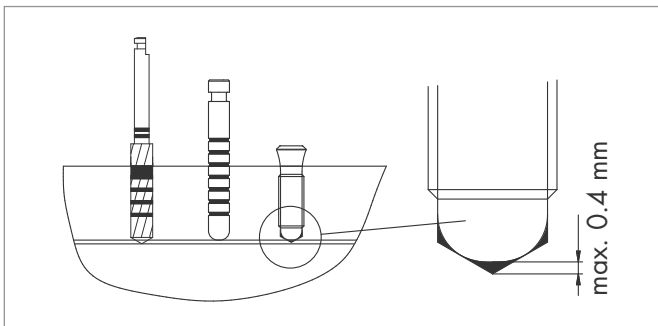

⚠ Warning


Due to the construction and function of the drills, the drill tip is a maximum of 0.4 mm longer than the implant insertion depth. This additional length must be taken into consideration during the planning phase.

3.2.4 PLANNING THE IMPLANT ANGLULATION USING THE STRAUMANN® PRO ARCH GUIDE

The Straumann® Pro Arch Guide is used for visual and three-dimensional orientation of the implant angulation (mesial/distal) and oral parallelization. The Straumann® Pro Arch Guide is used in edentulous jaws for surgical implant placement.

The template of the Straumann® Pro Arch Guide can be easily bent to adapt to the dental arch. It is secured by drilling into the symphysis with a \varnothing 2.2 mm Pilot Drill and a pin in the jaw. The drilling depth for the bone cavity of the pin is 10 mm. The drilling depth can be checked optically using the depth markings on the drills or using the optional depth stop system.

NC			RC		
Material	TAN	TAN	Material	TAN	TAN
Angle	17°	30°	Angle	17°	30°
\varnothing (mm)	4.6 mm	4.6 mm	\varnothing (mm)	4.6 mm	4.6 mm
GH (mm)	2.5 mm	2.5 mm	GH (mm)	2.5 mm	2.5 mm
	4.0 mm	4.0 mm		4.0 mm	4.0 mm

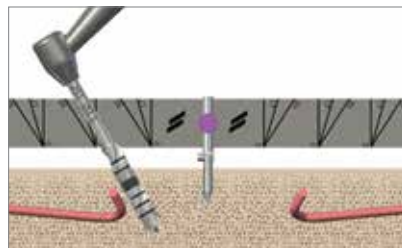

Product description

The Straumann® Pro Arch Guide is used in edentulous jaws for surgical implant placement. The template of the Straumann® Pro Arch Guide can be easily bent to adapt to the dental arch. It is secured by drilling into the symphysis with a \varnothing 2.2 mm Pilot Drill and a pin in the jaw. The drilling depth for the bone cavity of the pin is 10 mm. The drilling depth can be checked optically using the depth markings on the drills or using the optional depth stop system.

The slider is used to position the template for drilling. Drill the implant sites according to the surgical protocol. Each drill is aligned parallel to the template surface and at the implantation angle. Make sure the Guide is properly assembled, clean and sterile. Never use potentially contaminated components.

Warnings and precautions:

The following precautions are to be met prior to or during treatment:

- Position the patient in such a way that the danger of aspirating components is minimized. All components that are used intraorally must be secured against aspiration or swallowing.
- Do not use damaged or blunt instruments. Always inspect instruments before use.
- If the laser markings are illegible, the device must be replaced.
- Do not use more than 20 times.

For guidelines for the sterilization of the Straumann® Pro Arch Guide, please refer to the brochure *Surgical and Prosthetic Instruments, Care and Maintenance* (NAMLIT 1055).

4. Surgical procedures

4.1 IMPLANT BED PREPARATION

The implant diameter, implant type, position and number of implants should be selected individually taking the anatomy and spatial circumstances into account. The specific measurements should be regarded as minimum guidelines.

Note that for Straumann® Bone Level Tapered Implants the basic implant bed preparation depends on bone class. Please consult the brochure Straumann® Bone Level Tapered Implant, Basic Information NAMLIT 1043

Steps	Instrumentation
1. Basic implant bed preparation	
Ridge preparation	Round Bur (all implants)
Twist drilling	Pilot Drill 1 (2.2 mm) Alignment Pin Pilot Drill 2 (2.8 mm) Depth Gauge
2. Final implant bed preparation	
Profile drilling	Profile Drill BL/NNC Profile Drill TE Profile Drill BLT Profile Drill
Tapping	S/SP Tap BL/TE/NNC Tap BLT Tap

Basic implant bed preparation involves ridge preparation and twist drilling. For Twist Drilling, the endosteal diameter of the implant (3.3/4.1/4.8 mm) – not the implant type or the bone class – determines which instruments have to be used.


Final implant bed preparation involves profile drilling and tapping. For tapping, the implant type (S/SP/TE/BL/BLT) and the bone class determine which instruments have to be used, with exception of the NNC Implant that require a BL Tap.

Please note: Narrow Neck Crossfit and SP Short Implants have a Standard design, but requires a Bone Level tapping.

All Straumann dental implants are placed using one instrument kit – the Straumann® Surgical Cassette. The implant bed preparation covers two main steps.

4.1.1 BASIC IMPLANT BED PREPARATION

After opening the gingiva, the basic implant bed preparation begins with preparing the alveolar ridge (Step 1) and marking the implantation site with a Round Bur (Step 2). After that, the implant bed preparation with Pilot Drills and Twist Drills follows (Step 3–7), according to the endosteal implant diameter chosen in the preoperative planning (see Chapter 3).

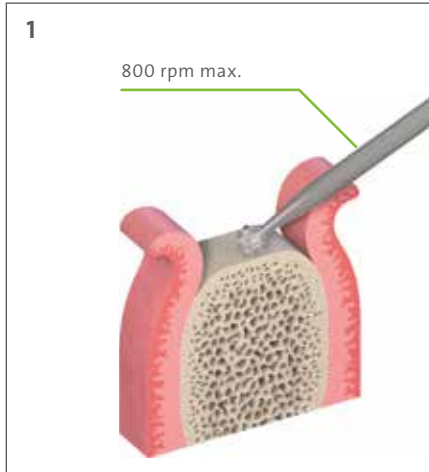

Step 1 – Prepare the alveolar ridge

Carefully reduce and smooth a narrow tapering ridge with a large Round Bur. This will provide a flat bone surface and a sufficiently wide area of bone.

Note


When choosing the implant length (SLActive®/SLA® surface), the vertical reduction of the bone has to be considered.



Step 2 – Mark the implantation site

Mark the implantation site determined during the implant position planning with the $\varnothing 1.4$ mm Round Bur or the $\varnothing 1.6$ Needle Drill. The Implant Distance Indicator can be used for that purpose (see pages 21–22). If the Distance Indicator is used together with the Needle Drill to mark the implant position, make sure not to drill more than 3 mm in order to avoid any collision between the Needle Drill and the Distance Indicator.

Widen and correct the position of the mark with the $\varnothing 2.3$ mm or the $\varnothing 3.1$ mm Round Bur, if necessary.

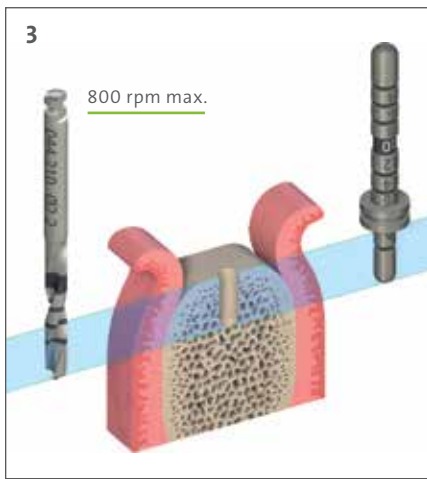

Step 3 – Mark the implant axis


With the $\varnothing 2.2$ mm Pilot Drill, mark the implant axis by drilling to a depth of about 6 mm.

Insert the short side of the Depth Gauge with the Implant Distance Indicator to check for correct implant axis orientation.

If necessary, correct unsatisfactory implant axis orientation in the following step.

Note

The Implant Distance Indicator visualizes the shoulder diameter of 4.8 mm (RN) and enables checking of the probable position of the implant shoulder.

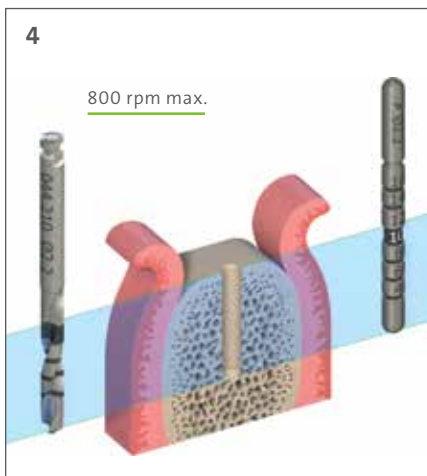


Step 4 – Prepare the implant bed to $\varnothing 2.2$ mm

Pre-drill the implant bed to the final preparation depth with the $\varnothing 2.2$ mm Pilot Drill.

Use the $\varnothing 2.2$ mm Alignment Pin to check the implant axis and preparation depth.

Caution

At this point take an X-ray, particularly with vertically reduced bone availability. The Alignment Pin is inserted into the drilled area, which allows a comparative visualization of the drill hole in relation to the anatomical structures.

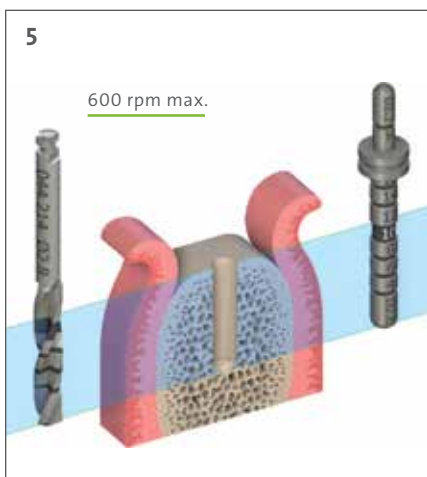


Step 5 – Widen the implant bed to $\varnothing 2.8$ mm

Continue with the implant bed preparation.

If necessary, correct the implant position with the $\varnothing 2.8$ mm Pilot Drill. Use the $\varnothing 2.8$ mm Depth Gauge to check the preparation depth.

For an implant with an endosteal diameter of 3.3 mm, basic preparation ends here. Continue with the fine implant bed preparation on page 31.

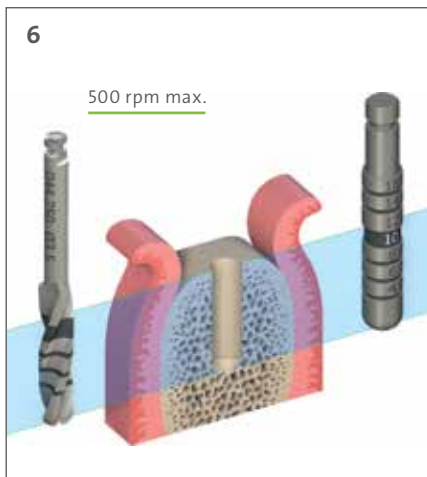


For $\varnothing 4.1$ mm and $\varnothing 4.8$ mm implants

Step 6 – Widen the implant bed to $\varnothing 3.5$ mm

Continue with the $\varnothing 3.5$ mm Straumann® Twist Drill PRO and check the final preparation depth with the $\varnothing 3.5$ mm Depth Gauge.

For an implant with an endosteal diameter of 4.1 mm, basic preparation ends here. Continue with the fine implant bed preparation on page 31.

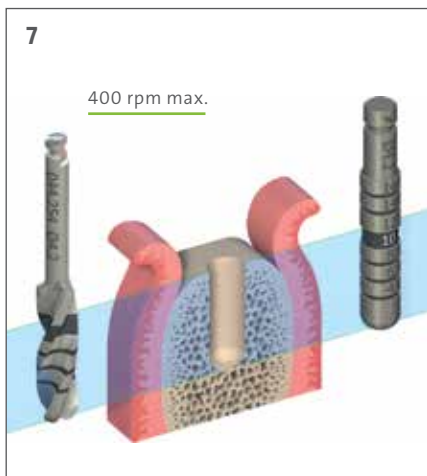

For $\varnothing 4.8$ mm implants

Step 7 – Widen the implant bed to $\varnothing 4.2$ mm
















Continue with the $\varnothing 4.2$ mm Straumann Twist Drill PRO and check the final preparation depth with the $\varnothing 4.2$ mm Depth Gauge.

Continue with the fine implant bed preparation on page 31.

Note

To facilitate introducing the instruments into the bone cavity, the bony margin of the drill hole can be beveled slightly using a large Round Bur or with an SP Profile Drill corresponding to the diameter of the Twist Drill/Spiral Drill employed. The Profile Drills are inserted only a fraction into the drill hole.

The following table summarizes the use of instruments for the basic implant bed preparation according to the endosteal implant diameter. All drills are available in a short and a long version and as multi-use as well as single-patient drills (see also Chapter 6.1). The table lists the short multi-use drills only.

Instrumentation for basic implant bed preparation					Endosteal \varnothing (mm)		
Step	Art. No.	Product	max. rpm		\varnothing 3.3	\varnothing 4.1	\varnothing 4.8
1 Preparation	044.004	Round Bur, \varnothing 3.1 mm	800		↓	↓	↓
	026.0054	Needle Drill, \varnothing 1.6 mm					
2 Mark implant position	044.022	Round Bur, \varnothing 1.4 mm	800		↓	↓	↓
	044.003	Round Bur, \varnothing 2.3 mm					
	044.004	Round Bur, \varnothing 3.1 mm					
3 Mark implant axis	044.210	Pilot Drill 1, short, \varnothing 2.2 mm	800		↓	↓	↓
	046.455	Depth Gauge, with Implant Distance Indicator, \varnothing 2.2/2.8 mm					
4 Prepare implant bed to \varnothing 2.2 mm	044.210	Pilot Drill 1, short, \varnothing 2.2 mm	800		↓	↓	↓
	046.458	Alignment Pin, \varnothing 2.2 mm, straight					
5 Prepare implant bed to \varnothing 2.8 mm	044.214	Pilot Drill 2, short, \varnothing 2.8 mm	600		↓	↓	↓
	046.455	Depth Gauge, with Implant Distance Indicator, \varnothing 2.2/2.8 mm					
6 Prepare implant bed to \varnothing 3.5 mm	044.250	Twist Drill PRO, short, \varnothing 3.5 mm	500			↓	↓
	046.450	Depth Gauge \varnothing 3.5 mm					
7 Prepare implant bed to \varnothing 4.2 mm	044.254	Twist Drill PRO, short, \varnothing 4.2 mm	400				↓
	046.451	Depth Gauge \varnothing 4.2 mm					

4.1.2 FINAL IMPLANT BED PREPARATION









The final implant bed preparation encompasses profile drilling and subsequent tapping. Instrumentation depends on the implant type, the endosteal implant diameter, and the bone class.

Profile drilling

The Profile Drill prepares the implant bed for a specific Straumann® implant.

- Straumann Standard Plus, Tapered Effect, and Bone Level/NNC Implants require profile drilling with specific instruments. This is independent of the bone class
- Straumann Standard Implants are inserted without profile drilling

The Profile Drills are clearly marked SP, TE, or BL/NNC. The (first) diameter indicated on the label corresponds to the diameter of the guide cylinder and, accordingly, to the diameter of the implant bed before profile drilling. All Straumann Profile Drills are available in a short and a long version.

Straumann Standard Plus Profile Drill	Straumann Tapered Effect Profile Drill	Straumann Bone Level Profile Drill
<div style="display: flex; justify-content: space-around;"> <div style="text-align: center;"> <p>Insertion depth on SLActive®SLA® surface margin level</p>  </div> <div style="text-align: center;"> <p>Insertion depth on implant shoulder</p>  </div> </div>	<div style="display: flex; justify-content: space-around;"> <div style="text-align: center;"> <p>Insertion depth on SLActive/SLA surface margin level</p>  </div> <div style="text-align: center;"> <p>Insertion depth on implant shoulder</p>  </div> </div>	<div style="display: flex; justify-content: space-around;"> <div style="text-align: center;">  </div> <div style="text-align: center;">  </div> </div>
<p>Insert the Straumann Standard Plus Profile Drill according to the planned insertion depth of the implant.</p>	<p>Insert the Straumann Tapered Effect Profile Drill according to the planned insertion depth of the implant.</p>	<p>Insert the Straumann Bone Level Profile Drill up to the planned implant shoulder level.</p> <p>A dent on the front of the guide cylinder makes the drills better distinguishable from Tapered Effect Profile Drills.</p>
<p>400 rpm max.</p>	<p>300 rpm max.</p>	<p>300 rpm max.</p>
Straumann Narrow Neck CrossFit® Profile Drill		
<div style="display: flex; justify-content: space-around;"> <div style="text-align: center;"> <p>Insertion depth on SLActive® surface margin level</p>  </div> <div style="text-align: center;"> <p>Insertion depth on implant shoulder</p>  </div> </div>		
<p>Insert the Straumann® Bone Level profile drill according to the planned insertion depth of the implant</p>		
<p>300 rpm max.</p>		

Note

Due to the unflared neck portion, Standard Plus Ø4.8 mm RN Implants are inserted without profile drilling.

Caution

The Profile Drills are suitable only for the corresponding implant type.

Tapping

Tapping prepares the implant bed for a specific thread type. It is an optional step that gives the surgeon the flexibility to adjust the surgical protocol to the bone class to help achieve optimal primary stability. It is recommended in dense bone and with large diameter implants in order to keep the insertion torque in a desirable range. The table below summarizes suggested tap usage.

Note

TE Implants generally do not need tapping.

In specific situations of TE Implants (e.g. dense bone conditions), the BL/TE/NNC Tap can be used according to the recommendation for BL Implants as suggested in the table below.

Tapping according to bone class

Bone Classes*	S, SP Implants		
	Endosteal diameter		
	Ø3.3 mm	Ø4.1 mm	Ø4.8 mm
Class 1	full	full	full
Class 2	coronal	coronal	full
Class 3			full
Class 4			full

Bone Classes*	BL/NNC/SPS Implants		
	Endosteal diameter		
	Ø3.3 mm	Ø4.1 mm	Ø4.8 mm
Class 1	full	full	full
Class 2	full	full	full
Class 3			full
Class 4			full

*Class 1: hardest bone/Class 4: Soft bone

coronal = thread tapping in the coronal area of the implant bed

full = thread tapping over full depth of the implant bed

Straumann® Standard and Standard Plus Taps

Tap for Adapter




S/SP Taps are used in the coronal area only or over the full depth of the implant bed, depending on implant diameter and bone class (see table above).

15 rpm max.

Straumann Bone Level and Tapered Effect Taps

Tap for Adapter

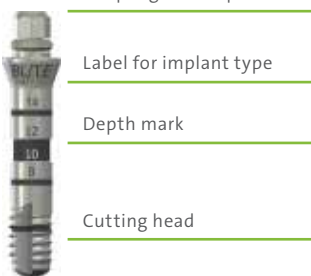

If a BL/TE Tap is used, it should always be inserted over the full depth of the implant bed preparation (see table above).

BL/TE Taps are available for Adapter only.

15 rpm max.

Caution

Straumann Taps are to be used only for the corresponding implant type.

Please note: For NNC and SPS Implants, the corresponding BL/TE/NNC and BL/TE Taps must be used.

Two types of Straumann® Taps are available: Taps for Ratchet and taps for Adapter. The Taps for Ratchet are directly coupled to the Ratchet, and are for tapping with Ratchet only. The Taps for Adapter can be coupled either to a Handpiece or a Ratchet Adapter and allow both, tapping with the Handpiece or with the Ratchet.

Tapping with Handpiece

Connect the Tap for Adapter to the Handpiece via the Handpiece Adapter. Do not exceed 15 rpm.




Tapping with Ratchet

For tapping with the Ratchet connect a Ratchet Adapter to the Tap for Adapter. After inserting the Tap into the cavity, the Ratchet is placed on its coupling and the thread is tapped with a slow rotating movement. The Holding Key is used as a stabilizer to maintain the direction of tapping during the procedure.




4.1.3 EXAMPLES FOR FINE IMPLANT BED PREPARATION: STRAUMANN® STANDARD AND STANDARD PLUS IMPLANTS

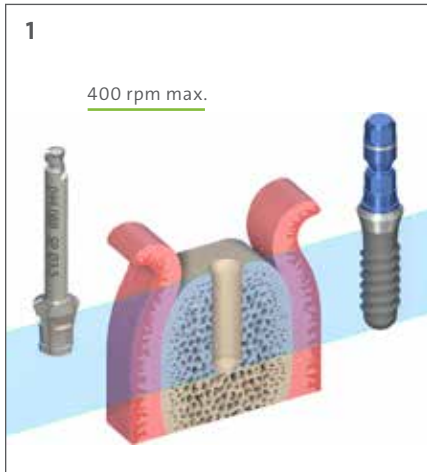

Step 1 – Standard Plus Profile Drill

Shape the coronal part of the implant bed with the Standard Plus Profile Drill.

Insert the Standard Plus Profile Drill up to the planned implant shoulder level (see page 31).

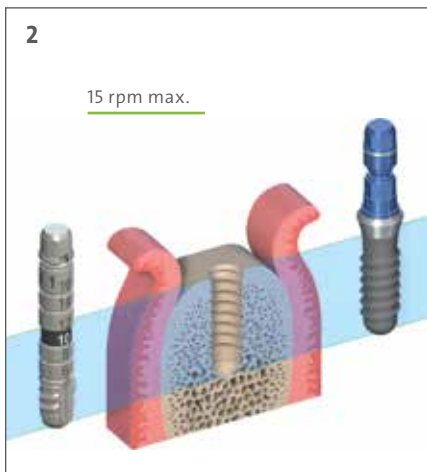


Step 2 – Tapping the thread in dense bone

Pre-tap the implant bed with the S/SP Tap according to the bone class and the endosteal diameter (see table on page 32).

Note

For Standard Implants, profile drilling is not required.

Please note that 4 mm SPS Implants and NNC Implants require the BL/TE and BL/TE/NNC Taps respectively.

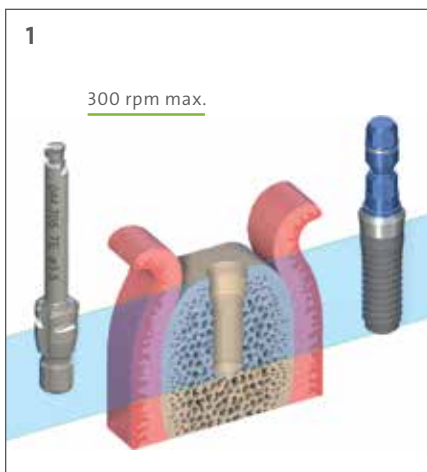

For Straumann Tapered Effect implants

Step 1 – TE Profile Drill

Shape the coronal part of the implant bed with the TE Profile Drill.

Insert the TE Profile Drill up to the planned implant shoulder level (see page 31).


Note

TE Implants generally do not need tapping.

In specific situations of TE Implants (e.g. dense bone conditions), the BL/TE Tap can be used according to the recommendation for BL Implants.


Straumann® Bone Level Implants

The following example shows fine implant bed preparation for a $\varnothing 4.1$ mm Bone Level Implant of 12 mm of length placed in bone class 1 or 2, making pre-tapping necessary (see table on page 32). These steps follow the basic implant bed preparation (see Chapter 4.1.1).

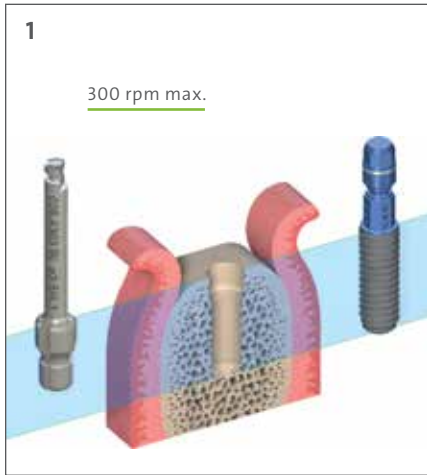

Step 1 – Bone Level Profile Drill



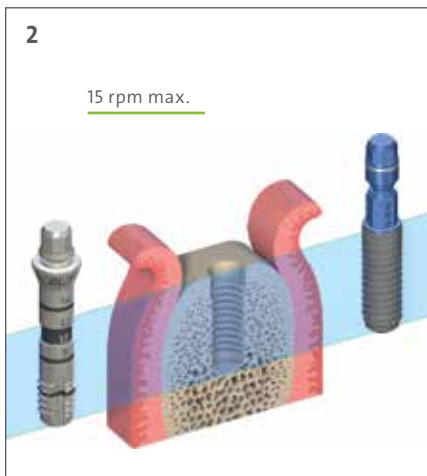






Prepare the implant bed with the Straumann Bone Level Profile Drill. Insert the Profile Drill up to the planned implant shoulder level (see page 31).



Step 2 – Tapping the thread in dense bone

Pre-tap the entire length of the implant bed with the BL/TE Tap.

The following table summarizes the use of Profile Drills and Taps for the fine implant bed preparation for all Straumann® implants. All Profile Drills are available in a short and a long version. S/SP Taps are available for Ratchet and for Adapter. The table lists the short Profile Drills, and the Taps for Adapter only.

Instrumentation for fine implant bed preparation					Straumann Standard Implant			
Art. No.	Product	Max. rpm		Thread pitch				
					S Ø 3.3 RN	S Ø 4.1 RN	S Ø 4.8 RN	S Ø 4.8 WN
044.086	SP Profile Drill, short, Ø 2.8 mm, RN	400						
044.088	SP Profile Drill, short, Ø 3.5 mm, RN							
044.084	SP Profile Drill, short, Ø 4.2 mm, WN							
044.575	S/SP Tap, Ø 3.3 mm, for Adapter	15		1				
044.577	S/SP Tap, Ø 4.1 mm, for Adapter			1.25				
044.579	S/SP Tap, Ø 4.8 mm, for Adapter			1.25				
044.701	TE Profile Drill, short, Ø 2.8 mm RN	300						
044.705	TE Profile Drill, short, Ø 3.5 mm RN							
044.703	TE Profile Drill, short, Ø 4.2 mm WN							
026.2303	BL Profile Drill, Ø 3.3 mm, short	300						
026.4303	BL Profile Drill, Ø 4.1 mm, short							
026.6303	BL Profile Drill, Ø 4.8 mm, short							
026.231	BL/TE/NNC Tap, Ø 3.3 mm, for Adapter	15		0.8				
026.431	BL/TE/NNC Tap, Ø 4.1 mm, for Adapter			0.8				
026.631	BL/TE/NNC Tap, Ø 4.8 mm, for Adapter			0.8				

- Required step
- More than 1 step required
- Required in dense bone only

* Due to the unflared neck portion, the Straumann® Standard Plus Ø4.8 mm RN Implants are inserted without profile drilling

Straumann Standard Plus Implant								Straumann Tapered Effect Implant			Straumann Bone Level Implant		
<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input type="checkbox"/>	<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input type="checkbox"/>	<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>
<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input checked="" type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input checked="" type="checkbox"/>

4.2 OPENING THE IMPLANT PACKAGE




Sterile barrier system: Blister

Step 1 – Open the blister and remove the vial

Note

The blister ensures the sterility of the implant. Do not open the blister until immediately prior to implant placement.

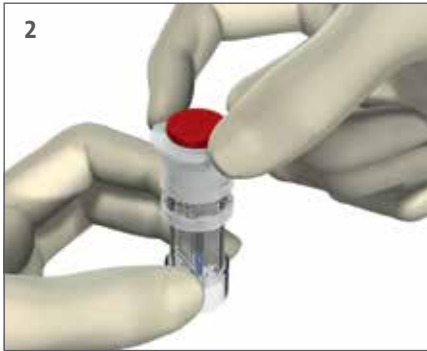


Step 2 – Open the vial

Turn the lid in counterclockwise direction.

SLActive® only: Keep the vial upright to prevent the liquid from flowing out.

Note

If the implant carrier is not firmly attached to the lid, screw in the lid once again.

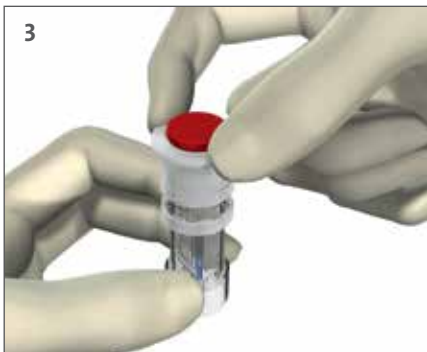

Step 3 – Detach the implant carrier

Detach the implant carrier from the lid by pulling it off manually.

Note (for SLActive only)

After removing the implant from the solution, the chemical activity of SLActive is ensured for 15 minutes.



Sterile barrier system: Vial

Step 1 – Open the safety cap

Open the safety cap of the sterile vial.

Note

The vial ensures the sterility of the implant.



Step 2 – Remove the implant from the carrier

Simultaneously, pull down the implant carrier and lift the implant out of the implant carrier (while supporting your arms).

4.3 PLACING THE IMPLANT

A Straumann implant can be placed either manually with the Ratchet or with the aid of the Handpiece. A maximum speed of 15 rpm is recommended. The following step-by-step instructions show how a Straumann® Standard Plus Implant is placed with the Handpiece (left column on the following pages) and how a Straumann Bone Level Implant is placed with the Ratchet (right column).

Note

Straumann Bone Level Implants must be rotationally oriented for both, Handpiece and Ratchet insertion (see Step 5 on page 43). Apart from this exception, all Straumann implants are placed in the same way.

Implant placement with Handpiece



Step 1 – Attach the Handpiece Adapter
Hold the enclosed part of the implant carrier. Attach the Handpiece Adapter to the Loxim®. A click will be heard when the Adapter is attached correctly.



Step 2 – Remove the implant from the carrier
Simultaneously, pull down the implant carrier and lift the implant out of the implant carrier (keep your arms steady).

Implant placement with Ratchet



Step 1 – Attach the Ratchet Adapter
Hold the enclosed part of the implant carrier. Attach the Ratchet Adapter to the Loxim. A click will be heard when the Adapter is attached correctly.



Step 2 – Remove the implant from the carrier
Simultaneously, pull down the implant carrier and lift the implant out of the implant carrier (keep your arms steady).



Step 3 – Place the implant

Place the implant with the Handpiece into the implant bed. Move the implant into its final position with a maximum of 15rpm turning it clockwise.


⚠ Caution

Vertical position corrections using reverse rotations (counterclockwise) may lead to a decrease in primary stability.

⚠ Caution

An insertion torque of 35Ncm is recommended. If 35Ncm is achieved before the implant has assumed its final position, to avoid bone overcompression, check that the implant bed preparation is correct.

The Loxim® is provided with a pre-determined breaking point to prevent damage to the inner configuration of the implant, thus ensuring the integrity of the interface for mounting the prosthesis.

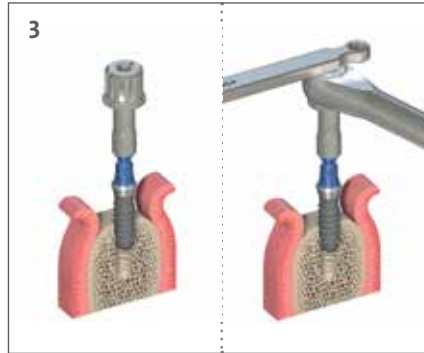

Step 3 – Place the implant


Place the implant with the Ratchet into the implant bed. Move the implant into its final position with a maximum of 15rpm turning it clockwise.

⚠ Caution

Vertical position corrections using reverse rotations (counterclockwise) may lead to a decrease in primary stability.

Step 4 – Not needed for S/SP/TE

S, SP, and TE Implants do not need to be rotationally oriented. If you are placing a Bone Level Implant with the Handpiece, choose the correct position as shown in Step 4 in the right column.

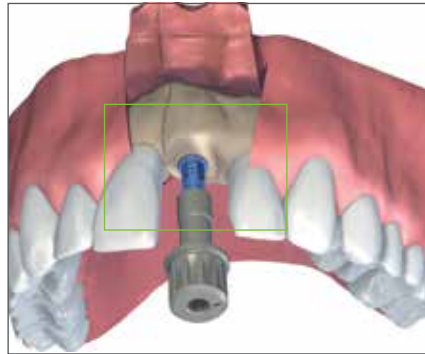


Step 4 – For BL/NNC Correct implant orientation

While approaching the final implant position, make sure that the drilled holes on the blue transfer part are oriented exactly orofacially. This positions the four protrusions of the internal connection for ideal prosthetic abutment orientation. A quarter turn to the next drilled holes corresponds to a vertical displacement of 0.2mm. The drilled holes also show the depth of the implant shoulder in the bone.

⚠ Caution


Avoid vertical position corrections using reverse rotations (counterclockwise). This can cause loosening of the transfer part and may lead to a decrease in primary stability.

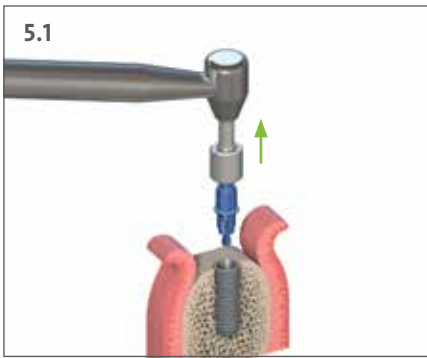

Step 5.1 – Remove the instruments with Loxim®

Loxim can easily be re-inserted to finish an uncompleted implant placement until the implant is fully inserted. If the implant needs to be removed during implantation surgery, Loxim allows for counter-clockwise turns.

After insertion, the Loxim is detached with the Adapter.

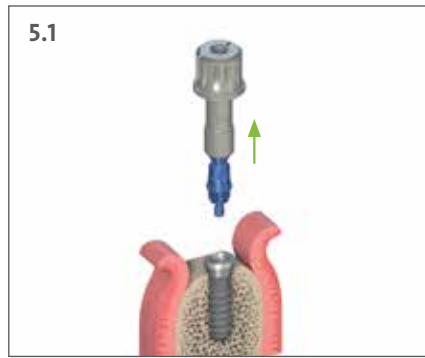

Step 5.1 – Remove the instruments with Loxim

Loxim can easily be re-inserted to finish an uncompleted implant placement until the implant is fully inserted. If the implant needs to be removed during implantation surgery, Loxim allows for counter-clockwise turns.

Remove the Ratchet while holding the Adapter at the bottom, and then detach the Adapter-Loxim assembly.




Step 5.2 – Loosen the transfer part without Loxim®

Before removing the transfer part, set the motor on the Handpiece to “reverse”. During the first few turns, hold the implant with the Holding Key which is used for stabilizing (countering) the hexagon. Remove the transfer part (for details of the Holding Key see Chapter 6.1.6).



Step 5.2 – Loosen the transfer part without Loxim

Change the direction of the Ratchet. The arrow on the rotary knob now points counterclockwise (see insert). Use the Holding Key to counter the octagon and loosen the transfer part counterclockwise using the Ratchet (for details of the Holding Key see Chapter 6.1.6).

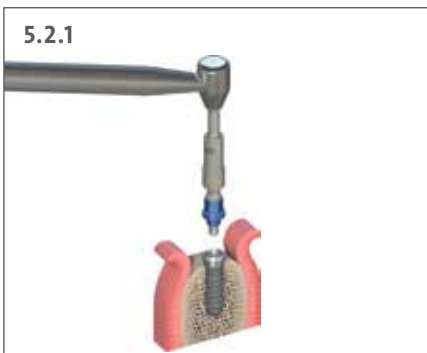

Step 5.2.1 – Remove the instruments

Remove the Holding Key and then completely remove the transfer part with the Adapter from the implant.



Step 5.2.1 – Remove the instruments

Remove the Holding Key, then the Ratchet, while holding the Adapter at the bottom. Finally, remove the transfer part from the implant with the Adapter still mounted completely.

⚠ Caution

In case of Loxim breakage, the remaining part of the Loxim in the implant must be removed. If the implant is not fitted correctly, it has to be unscrewed with a 48h Explantation Device. After that the implant bed should be re-prepared and a new implant be inserted. For further details, please consult the brochure *Guidance for Implant Removal*, USLIT 426 or 152.806.

4.3.1 ADDITIONAL INFORMATION FOR IMPLANTS WITH THE LOXIM® TRANSFER PIECE

Release aid for the Loxim Transfer Piece

For situations in which any removal force is to be avoided, a release aid for the Loxim can be used. Place the release aid onto the implant shoulder and hold it in place while detaching the Adapter with the Loxim.



Release aid S for Loxim® (for SC), 026.0073



Release aid N for Loxim® (for NNC and NC), 026.2558



Release aid R/W for Loxim® (for RC, RN and WN), 026.4558

Important additional information

An insertion torque of 35 Ncm is recommended. If 35 Ncm are achieved before the implant has reached its final position, make sure the implant bed preparation is correct to avoid bone overcompression.

Warning!

In case the implant has to be removed after implant placement, the retention of the Loxim in the implant may be reduced. Always secure the implant against aspiration when removing the implant.


The Loxim is provided with a pre-determined breaking point. If the Loxim breaks during implant insertion, one part remains in the Adapter and the other part in the implant. Both parts can be removed with tweezers.

To extract the implant if the Loxim breaks, simply take out the broken part of the Loxim from the Adapter and re-insert the Adapter on the Loxim part remaining in the implant. Counterclockwise turns will remove the implant.

The lower part of the Loxim is not secured in the Adapter and, additionally, needs to be secured against aspiration when taking out the implant.

Caution

The broken part of the Loxim no longer protects against high torque. Therefore, it is not to be used to advance the placement of the implant.

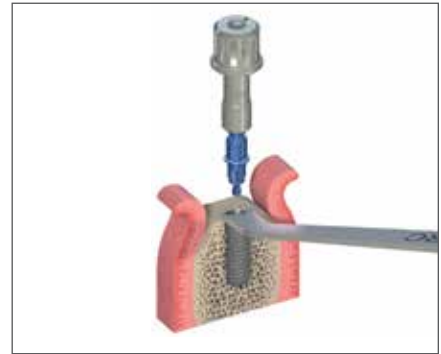


4.4 SOFT TISSUE MANAGEMENT

After implantation, the implant is closed – hand-tightened – with an SCS Closure Screw, Healing Cap or Healing Abutment to protect the implant (for SCS Screwdrivers see Chapter 6.1.7). The surgeon can choose between submucosal and transmucosal healing and has all options available for soft tissue management made possible through a set of secondary healing components.

4.4.1 SUBMUCOSAL HEALING

For submucosal healing (healing under closed mucoperiosteal flap) the use of a Closure Screw, shorter Healing Cap or Healing Abutment is recommended. Submucosal healing is suggested in esthetic indications and for implantations with simultaneous guided bone restoration (GBR) or membrane technique. A second surgical procedure is required for uncovering the implant and insertion of the desired secondary component.

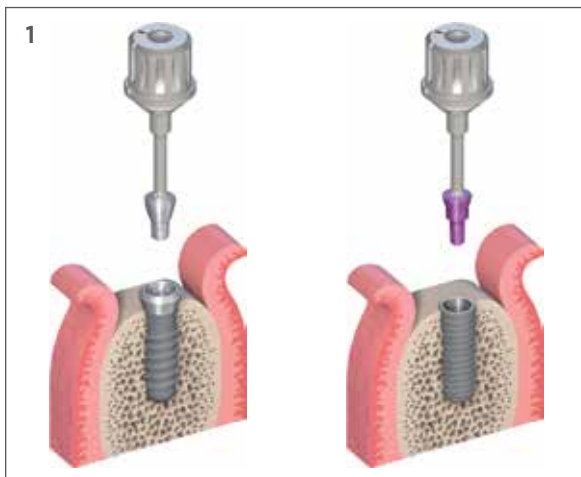

Step 1 – Inserting the Closure Screw after first surgery

Ensure that the internal configuration of the implant is clean and bloodless.

Pick up the Closure Screw with the SCS Screwdriver. The friction fit will secure the Closure Screw to the instrument during insertion and will allow safe handling.

Hand-tighten the Closure Screw. The design will provide a tight connection between the two components.

Note

All Closure Screws are delivered sterile and ready to use.


Subsequent loosening is made easier by applying chlorhexidine gel or sterile Vaseline to the Closure Screw before it is screwed into the implant.



Step 2 – Wound closure

Adapt the mucoperiosteal flaps carefully and suture together with interrupted sutures.

Make sure a tight seal is formed over the implant.

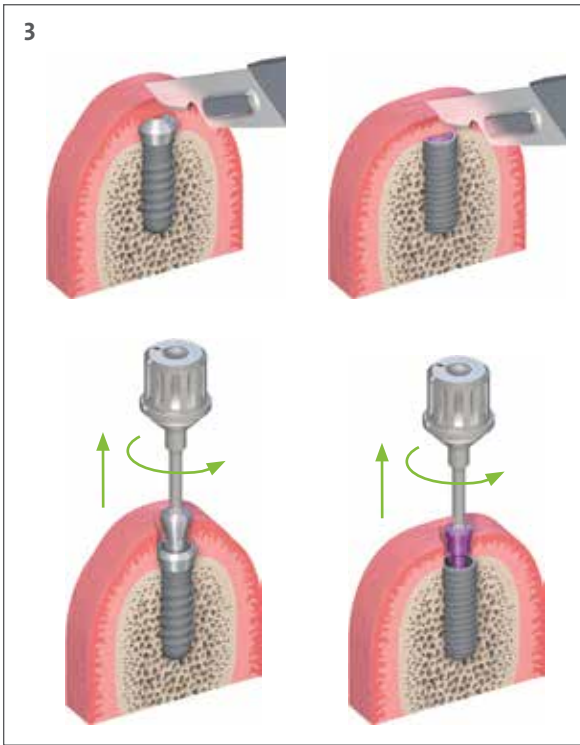


Step 3 – Reopening and removal: Second surgery

Locate the implant.

Make a small crestal incision down to the Closure Screw.

Spread the flap slightly and remove the Closure Screw with the SCS Screwdriver.

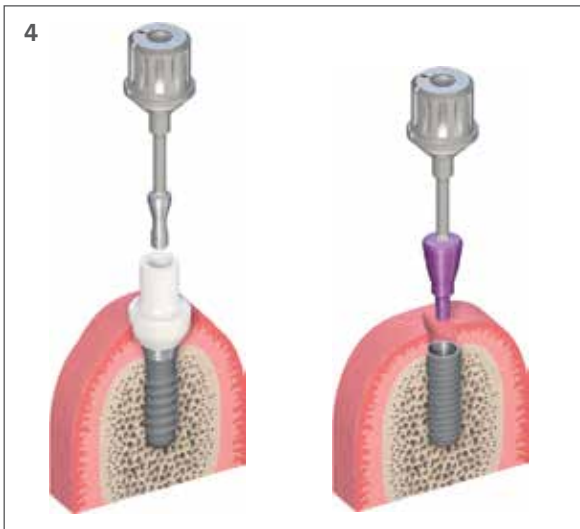

Step 4 – Insertion and wound closure


Rinse the exposed internal connection of the implant thoroughly with sterile saline solution.

Insert a suitable secondary component. (For optimal Bone Level Healing Abutment selection see Chapter 5.)

Adapt the soft tissue and suture it back tightly without tension around the secondary component.

4.4.2 TRANSMUCOSAL HEALING

A versatile portfolio of Healing Caps and Healing Abutments is available for all Straumann implants enabling soft-tissue sculpturing during transmucosal healing. They are recommended for intermediate use. After the soft-tissue healing phase they are replaced with the appropriate temporary or final restoration. (For optimal Bone Level Healing Abutment selection see Chapter 5)

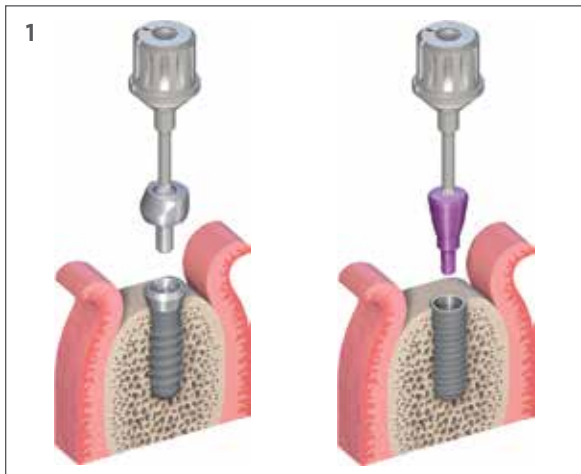


Step 1 – Insertion

Ensure that the internal configuration of the implant is clean and bloodless.

Insert the Healing Cap or Healing Abutment with the SCS Screwdriver. The friction fit secures the components to the instrument during insertion and ensures safe handling.

Hand-tighten the Healing Cap or Healing Abutment. The design will provide a tight connection between the two components.

Subsequent loosening is made easier by applying chlorhexidine gel or sterile Vaseline to the Healing Cap or Healing Abutment before they are screwed into the implant.

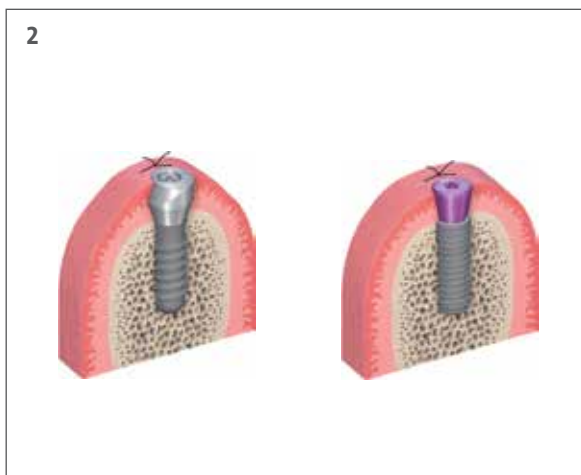

Step 2 – Wound closure

Adapt the soft tissue and suture it back tightly around the abutment.

OVERVIEW OF BONE LEVEL ABUTMENTS AND CORRESPONDING HEALING ABUTMENTS

Which Healing Abutments suit which abutments?

Cement-retained solutions

Platform		SC			
Abutment	Type	Screw- /cement-retained			
	Material	TAN			
	Angle	0°			
	Ø(mm)	Oval shape 3.3 x 4.3			
	GH (mm)	1	2		
Healing Cap	GH (mm)	1	2	3	
	Abutment height	1	3.5	5	6.5
	Ø(mm)	Oval shape 3.3 x 4.3			
	Material	Ti			
	Type	Conical healing Abutment			









Platform		NC													
Abutment	Type	Anatomic Abutment						Cementable Abutment							
	Material	Ti		Ti		IPS e.max®		IPS e.max		Ti			Ti		
	Angle	0°		15°		0°		15°		0°			0°		
	Ø(mm)	4.0		4.0		4.0		4.0		3.5			5.0		
	GH (mm)	2.0	3.5	2.0	3.5	2.0	3.5	2.0	3.5	1.0	2.0	3.0	1.0	2.0	3.0
Healing Cap	GH (mm)	3.5	5.0	3.5	5.0	3.5	5.0	3.5	5.0	3.5	3.5	5.0	3.5	3.5	5.0
	Ø(mm)	4.8				4.8				4.8			3.6		
	Type	NC Conical healing Abutment				NC Conical healing Abutment				NC Conical healing Abutment			NC Conical healing Abutment		









Platform		RC													
Abutment	Type	Anatomic Abutment						Cementable Abutment							
	Material	Ti		Ti		IPS e.max		IPS e.max		Ti			Ti		
	Angle	0°		15°		0°		15°		0°			0°		
	Ø(mm)	6.5		6.5		6.5		6.5		5.0			6.5		
	GH (mm)	2.0	3.5	2.0	3.5	2.0	3.5	2.0	3.5	1.0	2.0	3.0	1.0	2.0	3.0
Healing Cap	GH (mm)	4.0	6.0	4.0	6.0	4.0	6.0	4.0	6.0	2.0	4.0	6.0	2.0	4.0	6.0
	Ø(mm)	6.5				6.5				5.0			6.5		
	Type	RC Conical healing Abutment				RC Conical healing Abutment				RC Conical healing Abutment			RC Conical healing Abutment		

Screw-retained solutions

Platform		NC												
Abutment	Type	Anatomic Abutment				Screw-Retained Abutment								
	Material	IPS e.max®		IPS e.max		TAN			TAN			TAN		
	Angle	0°		15°		0°			17°			30°		
	∅(mm)	4.0		4.0		3.5	4.6		4.6			4.6		
	GH (mm)	2.0	3.5	2.0	3.5	1.0	2.5	4.0	2.5	4.0	5.5	2.5	4.0	5.5
Healing Cap	GH (mm)	3.5	5.0	3.5	5.0	2.0	3.5	5.0	2.0	3.5	5.0	2.0	3.5	5.0
	∅(mm)	4.8				3.6			4.8			4.8		
	Type													
		NC Conical healing Abutment												
Healing Abutment														
			NC Ceramic Healing abutments											
Platform		RC												
Abutment	Type	Anatomic Abutment				Screw-Retained Abutment								
	Material	IPS e.max		IPS e.max		Ti			Ti			Ti		
	Angle	0°		15°		0°			17°			30°		
	∅(mm)	6.5		6.5		4.6	4.6		4.6			4.6		
	GH (mm)	2.0	3.5	2.0	3.5	1.0	2.5	4.0	2.5	4.0	5.5	2.5	4.0	5.5
Healing Cap	GH (mm)	4.0	6.0	4.0	6.0	2.0	4.0	6.0	2.0	4.0	6.0	2.0	4.0	5.0
	∅(mm)	6.5				5.0			6.5			5.0		
	Type													
		RC Conical healing Abutment / RC Ceramic Healing abutments												
Healing Abutment														
			RC Ceramic Healing abutments											

Hybrid solutions

Platform		SC														
Abutment	Type	Screw-Retained Abutment									LOCATOR®					
	Material	TAN			TAN			TAN			Ti alloy					
	Angle	0°			17°			30°			0°					
	∅(mm)	3.5		4.6		4.6			4.6			3.8				
	GH (mm)	1.0	2.5	4.0	2.5	4.0	5.5	2.5	4.0	5.5	2.0	3.0	4.0	5.0	6.0	
	GH (mm)	2.0	3.5	5.0	2.0	3.5	5.0	2.0	3.5	5.0	2.0	3.5	5.0			
Healing Cap	∅(mm)	3.6			4.8			4.8			3.6					
	Type															
																
		NC Conical healing Abutment														

Platform		RC														
Abutment	Type	Screw-Retained Abutment									LOCATOR					
	Material	TAN			TAN			TAN			Ti alloy					
	Angle	0°			17°			30°			0°					
	∅(mm)	4.6			4.6			4.6			3.8					
	GH (mm)	1.0	2.5	4.0	2.5	4.0	5.5	2.5	4.0	5.5	1.0	2.0	3.0	4.0	5.0	6.0
	GH (mm)	2.0	4.0	4.0	2.0	3.5	5.0	2.0	3.5	5.0	2.0	4.0	6.0			
Healing Cap	∅(mm)	5.0			6.5			5.0			5.0					
	Type															
																
		RC Conical healing Abutment														

5. Healing Phase

5.1 HEALING PHASE DURATION

Situation	Healing phase	
	SLActive®	SLA®
<ul style="list-style-type: none"> • Good bone quality and adequate bone quantity • Implants with a diameter of 4.1 mm or 4.8 mm and a Straumann® SLActive/SLA surface length of ≥ 8 mm 	At least 3 – 4 weeks	At least 6 weeks
<ul style="list-style-type: none"> • Cancellous bone quality • Implants with a diameter of 2.9 mm • Implants with a diameter of 3.3 mm • Implants with a Straumann SLActive/SLA surface length of 6 mm 	At least 8 weeks	At least 12 weeks
<ul style="list-style-type: none"> • Straumann® Standard Plus Short Implant 	10 – 12 weeks	N/A
<ul style="list-style-type: none"> • Straumann SLActive/SLA surface is not completely in contact with the bone • Bone augmentation measures* are necessary 	Healing phase corresponding to the situation	

*This technique should be employed only by dentists who have adequate experience in the use of augmentation procedures.

6. Additional Information on Instruments

6.1 SURGICAL INSTRUMENTS

Instruments must be checked for completeness and function. An adequate stock of implants and spare sterile instruments should always be available. The instruments must be disassembled for sterilization. Well maintained instruments prevent infections from developing that could endanger patients as well as the practice team.

To avoid contamination of the operation field, all of the instruments and material employed must be sterile. To prevent contamination of the sterile instruments, they should be removed from the Surgical Cassette with the sterile Instrument Tweezers and put into the handle or Ratchet. The Instrument Tweezers (Art. No. 046.110) were developed and shaped specially to allow round instruments to be gripped securely.

All steps related to the maintenance of Straumann surgical instruments are part of a dental practice hygiene plan (see also *Care and Maintenance of Surgical and Prosthetic Instruments*, USLIT 119).

6.1.1 DEPTH MARKS ON STRAUMANN INSTRUMENTS

Straumann instruments have depth marks in 2 mm intervals that correspond to the available implant lengths. The marks on drills are continuous between 10 mm and 12 mm. The lower edge of the mark corresponds to 10 mm and the upper edge to 12 mm.

When inserting a Straumann® Standard Plus or Tapered Effect Implant up to the implant shoulder level (see Chapter 3), the preparation depth must be 2 mm deeper than the indicated implant length.


Example: The preparation depth for a 10 mm SP Implant inserted up to shoulder level must be 12 mm.

Warning


Due to the function and design of the drills, the drill tip is 0.4 mm longer than the insertion depth of the implant (see also Chapter 3.2.2.3).



Instrument Tweezers

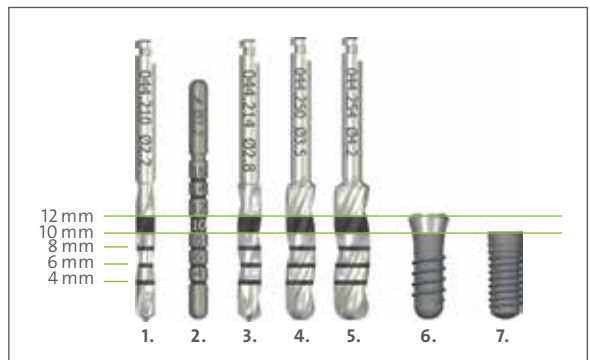

- 1. Pilot Drill 1, \varnothing 2.2 mm
- 2. Alignment Pin, \varnothing 2.2 mm
- 3. Pilot Drill 2, \varnothing 2.8 mm
- 4. Twist Drill PRO, \varnothing 3.5 mm
- 5. Twist Drill PRO, \varnothing 4.2 mm
- 6. Straumann Standard Plus Implant, \varnothing 4.1 RN, length 10 mm
- 7. Straumann Bone Level Implant, \varnothing 4.1 RC, length 10 mm

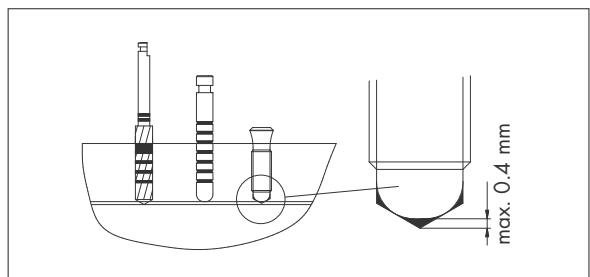

6.1.2 SINGLE-PATIENT DRILLS

Like multi-use drills, single-patient drills are indicated for the preparation of the implant bed for Straumann dental implants. They are supplied sterile and are to be used for one operation only and for one patient only. Single-patient drills can minimize the risk of infection for the patient. They are color-coded for easy identification of the diameter width.

Due to the function and design of the drills, the drill tip is 0.4 mm longer than the insertion depth of the implant.

For more information, please refer to *Straumann Single Patient Instruments, Surgical User Guide* (702173/en).

6.1.3 STRAUMANN® DRILL STOP – PRECISE DEPTH CONTROL


The Straumann® Drill Stop provides precise control over drilling depth during implant bed preparation for the placement of Straumann dental implants. Delivered in sterile sets the Drill Stops are ready to use. The Straumann® Drill Stop is designed for single-patient use only and must be used in conjunction with the single-patient drills specifically designed for them.

Note: Straumann® Drill Stops are not indicated for:

- Extraction sites, where bone cavity is often wider than the diameter necessary to hold the Drill Stop.
- Use with drill templates, due to the interference from or with the template.

6.1.4 STRAUMANN® SURGICAL CASSETTE

The Surgical Cassette is used for the secure storage and sterilization of the surgical instruments and auxiliary instruments of the Straumann Dental Implant System. The Surgical Cassette is made of a highly shock-proof thermoplastic, which has been proven for years in the medical area and is suitable for frequent sterilization in the autoclave.

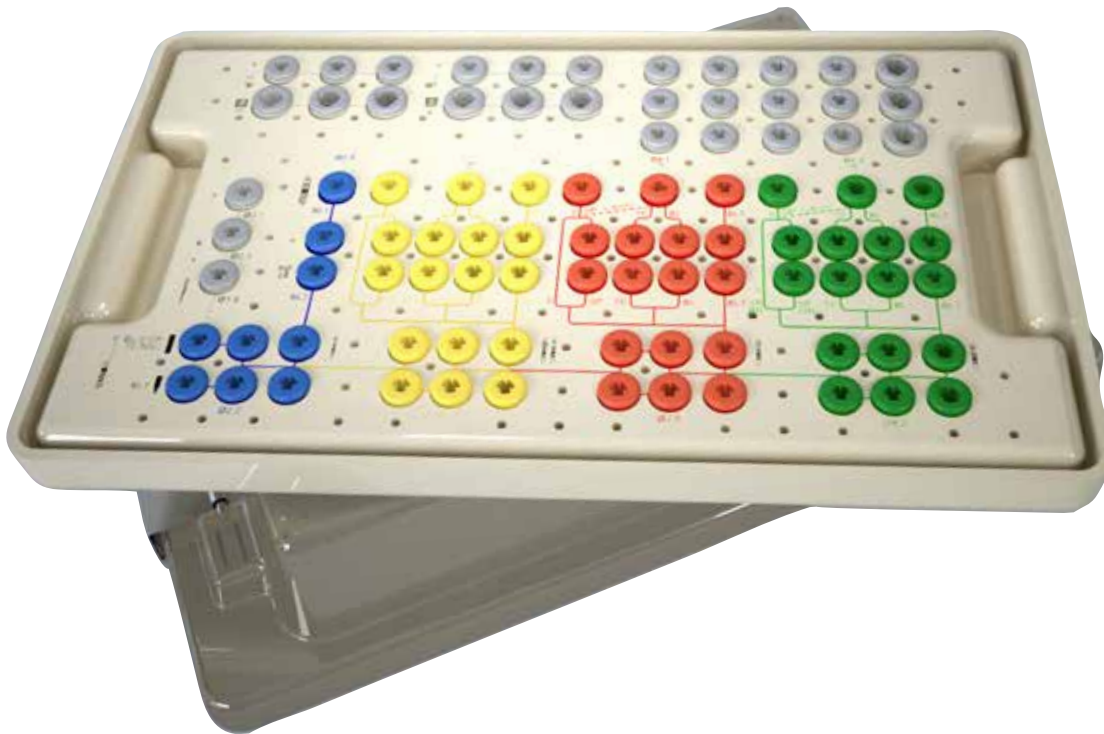

- Endosteal implant diameter 2.9 mm
- Endosteal implant diameter 3.3 mm
- Endosteal implant diameter 4.1 mm
- Endosteal implant diameter 4.8 mm



For guidelines for the sterilization of the Straumann Surgical Cassette, refer to the brochure Care and maintenance of surgical and prosthetic instruments, NAMLIT 1055.

6.1.5 RATCHET



Ratchet



Service Instrument



Ratchet disassembled

The Ratchet of the Straumann® Dental Implant System is a two-part lever arm instrument with a rotary knob for changing the direction of force.

The Ratchet is required for the following operations:

- Manual thread tapping
- Manual placement of implants into their final position in the implant bed

The Ratchet is supplied with a Service Instrument, which is used to loosen the headed screw.

After loosening, the Ratchet bolt can be removed from the body of the Ratchet. It must be disassembled for cleaning and sterilization.

6.1.6 HOLDING KEY

The Holding Key is used for:

- Stabilizing the Ratchet
- Countering the transfer part



Holding Key

Stabilizing the Ratchet

Use the pivot of the Holding Key to stabilize the Ratchet during implant insertion or during tapping.



Stabilizing the Ratchet

Countering the transfer part

Use the Holding Key for countering when loosening the transfer part from the implant. The transfer part should be loosened only with the Ratchet or Handpiece (counterclockwise).

The shape of the Holding Key is specially designed for different oral situations

- Forked end: When spaces are normal, the forked end is attached directly to the hexagon
- Closed end: When the interdental space is limited, the closed end must be placed on the hexagon over the transfer part. To do this, the Ratchet and Adapter or Handpiece must be removed



Forked end



Closed end

6.1.7 SCS SCREWDRIVERS



SCS Screwdriver for manual use

Article: Extrashort, short, long
Lengths: 15 mm, 21 mm, 27 mm
Material: Stainless steel



SCS Screwdriver for mechanical use in the Handpiece

Article: Extrashort, short, long
Lengths: 20 mm, 26 mm, 32 mm
Material: Stainless steel

6.2 OSTEOTOMES


6.2.1 Instrument set for bone condensation

- Indicated in cases with cancellous bone (bone class 3 and 4).
- Reinforces bone radially to give improved primary stability to the implant.
- Before the instruments are used, it is advisable to mount the depth stops in order not to exceed the pre-determined working depth. These are mounted onto the instrument using a SCS screwdriver.
- Instruments of increasing diameter are introduced manually using slightly rotary movements or, if necessary, lightly tapping with a hammer in accordance with the desired implant length and implant diameter.
- Insert the implant carefully without applying extra force.


Note: The instruments with diameters of 2.2 mm, 2.8 mm, 3.5 mm and 4.2 mm match the implant diameters of the Straumann® Dental Implant System. They are available as a straight or angled model, which facilitates access in the posterior region.



Osteotomes for bone condensation

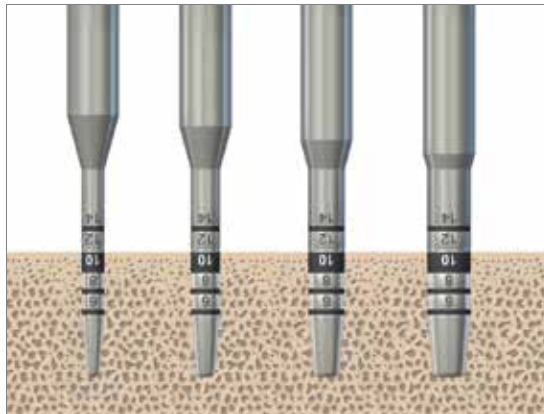

Insert Osteotomes to the desired implant length using slightly rotary movements.

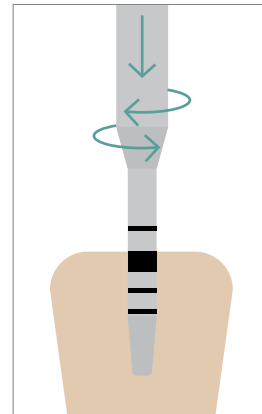

6.2.2 Instrument set for transalveolar sinus floor elevation


Indicated in cases with inadequate vertical bone. By tapping on the osteotomes with a mallet, the sinus floor can be fractured and elevated.

- The bone is prepared using the twist drills (\varnothing 2.2 mm/2.8 mm/3.5 mm/4.2 mm) in accordance with the desired implant diameter. The surgeon feels his or her way very carefully up to the cortical bone of the sinus floor (minimum distance 1 mm). This process requires precise radiological planning.
- Before the instruments are used, it is advisable to mount the depth stops in order not to exceed the pre-determined working depth. These are mounted onto the instrument using a SCS screwdriver.
- First, the sinus floor is fractured, which requires precise radiological planning. The use of depth stops is also recommended in order not to exceed the pre-determined working depth. The instrument is introduced by lightly tapping with a hammer in accordance with the desired implant length.
- During elevation, autologous and/or alloplastic filling or bone material should also be applied to the implant bed. The material introduced acts like a cushion lifting the mucous membrane in accordance with principles of hydraulics.
- Insert the implant carefully without applying extra force.

Note: The instruments with diameters of 2.2 mm, 2.8 mm, 3.5 mm and 4.2 mm match the implant diameters of the Straumann® Dental Implant System. They are available as a straight or angled model, which facilitates access in the posterior region.



Osteotomes for sinus floor elevation

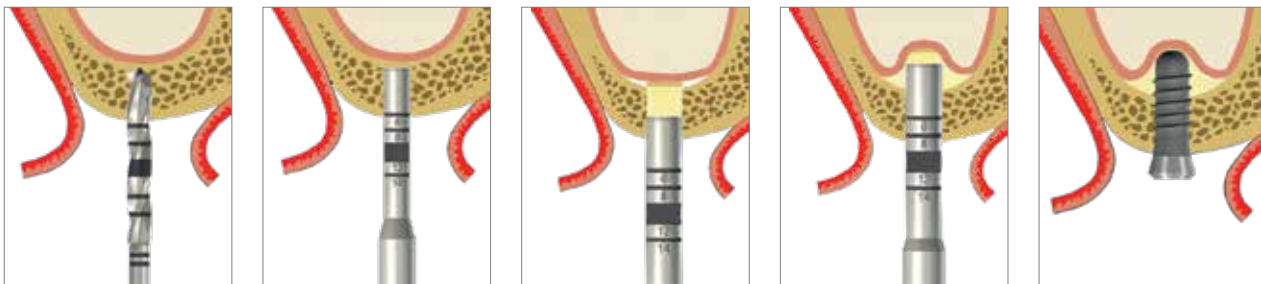

6.2.3 Depth stops for osteotomes

All osteotomes have clear laser marks for depths of 6 mm, 8 mm, 10 mm, 12 mm and 14 mm. In addition, adjustable depth stops facilitate depth checking.



Depth stops for osteotomes

6.3 CLEANING AND CARE OF INSTRUMENTS

Careful treatment of all instruments is of the utmost importance. Even slight damage for instance to the drill tips (e.g., when the drills are “thrown” into a bowl of water) impairs cutting performance and thus the clinical result. With correct and careful care, the high quality of the material and excellent workmanship ensure that the rotating instruments (drills*, taps etc.) can be used repeatedly (up to a maximum of ten times is recommended). *The Surgery Tracking Sheet for Straumann Cutting Instruments, NAMLIT 1051.* helps to give an overview of how often the individual instruments have already been used.

Instruments with high cutting performance are a basic requirement for successful implantation. The following should therefore be remembered:

- Never allow instruments to land on their tips
- Use each instrument only for its intended purpose
- Never let surgical residues (blood, secretion, tissue residues) dry on an instrument; clean immediately after surgery
- Thoroughly clean off incrustations with soft brushes only. Disassemble instruments, clean cavities especially well
- Never disinfect, clean (also ultrasound) or sterilize instruments made of different materials together
- Use only cleaning agents and disinfectants intended for the material and follow the instructions for use of the manufacturer
- Rinse disinfectants and cleaning agents very thoroughly with water
- Never leave or store instruments moist or wet

You will find detailed information in the brochure *Care and Maintenance of Surgical and Prosthetic Instruments, NAMLIT 1055.*

Surgery tracking sheet for Straumann cutting instruments (NAMLIT 1051)

Article	Art. No.	Dimensions	Number of surgical procedures																	
			1	2	3	4	5	6	7	8	9	10								
Round Bur	044.022	Ø1.4 mm																		
Round Bur	044.003	Ø2.3 mm																		
Round Bur	044.004	Ø1.1 mm																		
Pilot Drill 1	044.210	Ø2.2 mm, short																		
Pilot Drill 1	044.211	Ø2.2 mm, long																		
Pilot Drill 2	044.214	Ø2.8 mm, short																		
Pilot Drill 2	044.215	Ø2.8 mm, long																		
Twist Drill PRO	044.250	Ø1.5 mm, short																		
Twist Drill PRO	044.251	Ø1.5 mm, long																		
Twist Drill PRO	044.254	Ø4.2 mm, short																		
Twist Drill PRO	044.255	Ø4.2 mm, long																		
SP Profile Drill, KN	044.086	Ø2.8 mm, short																		
SP Profile Drill, KN	044.087	Ø2.8 mm, long																		
SP Profile Drill, KN	044.088	Ø1.5 mm, short																		
SP Profile Drill, KN	044.089	Ø1.5 mm, long																		
SP Profile Drill, WN	044.084	Ø4.2 mm, short																		
SP Profile Drill, WN	044.085	Ø4.2 mm, long																		
TE Profile Drill, KN	044.701	Ø2.8 mm, short																		
TE Profile Drill, KN	044.708	Ø2.8 mm, long																		
TE Profile Drill, KN	044.705	Ø1.5 mm, short																		
TE Profile Drill, KN	044.712	Ø1.5 mm, long																		
TE Profile Drill, WN	044.703	Ø4.2 mm, short																		
TE Profile Drill, WN	044.710	Ø4.2 mm, long																		
BL/N/C Profile Drill	026.2303	Ø3.3 mm, short																		
BL/N/C Profile Drill	026.2306	Ø3.3 mm, long																		
BL Profile Drill	026.4303	Ø4.1 mm, short																		
BL Profile Drill	026.4306	Ø4.1 mm, long																		
BL Profile Drill	026.6303	Ø4.8 mm, short																		
BL Profile Drill	026.6306	Ø4.8 mm, long																		
1/8" Tap for Ratchet	044.590	Ø3.3 mm, short																		
1/8" Tap for Ratchet	044.591	Ø3.3 mm, long																		

*Exception: “Single-patient drills“

7. Appendix

7.1 RELATED DOCUMENTATION

Note

Our detailed documentation will help you in carefully planning and performing your implant-based restorations:

- *Prosthetic Procedures for the Narrow Neck CrossFit® Implant – Straumann® Narrow Neck CrossFit Implant Line*, USLIT 408
- *Crown and Bridge Restorations: Straumann synOcta® Prosthetic System*, USLIT 187 or CALIT 187
- *Cement-retained Crowns and Bridges with the Solid Abutment System: Straumann Solid Abutment Prosthetic System*, USLIT 045
- *Basic Information on the Straumann Prosthetic Procedures – Straumann Bone Level*, USLIT 232 or CALIT232

Instrument care and maintenance

Well maintained instruments are a basic requirement for successful treatment. You will find detailed information in the brochure *Care and Maintenance of Surgical and Prosthetic Instruments*, NAMLIT 1055.

The Straumann Guarantee®

As a Swiss company, we attach the greatest importance to manufacturing our products in to the highest quality. We are firmly convinced of the scientific and clinical basis of our Straumann Dental Implant System and draw on the fund of know-how from nearly 30 years of quality production. The Straumann guarantee regulates replacement of all components of the Straumann Dental Implant System. You will find detailed information in the brochure *Straumann Guarantee*, USLIT 479 or CALIT 502.

Explanation

For explanation guidelines please refer to the Directions for Use: *Explanation Procedure for Straumann Dental Implants*, USLIT 426. The components required for explanation can be found in our current product catalog.

References

The Straumann Dental Implant System has been comprehensively clinically documented for over 25 years. You can find references to the current research literature on our website www.straumann.com or by contacting your local Straumann representative.

Courses and training

Continuing education ensures long-term success! Please, ask your Straumann representative directly for information on the Straumann Dental Implant System courses and training. Further information at www.straumann.us and www.straumann.ca.

7.2 IMPORTANT GUIDELINES

Please note

Practitioners must have appropriate knowledge and instruction in the handling of all Straumann Products for using the Straumann® Products safely and properly in accordance with the instructions for use.

The Straumann Product must be used in accordance with the instructions for use provided by the manufacturer. It is the practitioner's responsibility to use the device in accordance with these instructions for use and to determine, if the device fits to the individual patient situation.

The Straumann Products are part of an overall concept and must be used only in conjunction with the corresponding original components and instruments distributed by Straumann USA, LLC or Straumann Canada Limited, its ultimate parent company and all affiliates or subsidiaries of such parent company ("Straumann"), except if stated otherwise in this document or in the instructions for use for the respective Straumann Product. If use of products made by third parties is not recommended by Straumann in this document or in the respective instructions for use, any such use will void any warranty or other obligation, express or implied, of Straumann.

Availability

Some of the Straumann Products listed in this document may not be available in all countries.

Caution

In addition to the caution notes in this document, our products must be secured against aspiration when used intraorally.

Validity

Upon publication of this document, all previous versions are superseded.






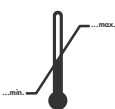








Documentation

For detailed instructions on the Straumann Products contact your Straumann representative.

Copyright and trademarks

Straumann documents may not be reprinted or published, in whole or in part, without the written authorization of Straumann. Straumann and/or other trademarks and logos from Straumann mentioned herein are the trademarks or registered trademarks of Straumann Holding AG and/or its affiliates.

Explanation of the symbols on labels and instruction leaflets

	Batch code
	Catalogue number
	Sterilized using irradiation
	Lower limit of temperature
	Upper limit of temperature
	Temperature limitation
Rx only	Caution: U.S. federal law restricts this device to sale by or on the order of a dental professional.
	Do not re-use
	Non-sterile
	Caution, consult accompanying documents
	Use by
	Keep away from sunlight
	Straumann Products with the CE mark fulfill the requirements of the Medical Devices Directive 93/42 EEC
	
	Consult instructions for use ifu.straumann.com

8. Index.

axial orientation	11
bone	
availability	12, 21, 24, 28
augmentation	16, 52
class	26, 31, 32, 34, 35, 58
quality	52
quantity	52
buccal wall	18
cemento-enamel-junction	17
color-coded	54, 6
contraindication	8
contralateral	17
coronal	4, 32, 34
coronoapical	4, 11, 17
CrossFit® connection	4–6, 8, 20, 26, 61
dental practice hygiene plan	53
depth marks	53
Diagnostic T	14, 19
drill template	
custom-made	11, 22
single tooth gaps	13, 19
vacuum-formed	21, 22
drilling depth	25, 54
endosteal diameters	3, 9, 10, 26–32, 34, 55
full-depth tapping	32
healing phase duration	48, 52
Holding Key	33, 44, 57
implant	
diameter	3, 9, 10, 12, 26, 27, 30–32, 52
Implant Distance Indicator	20, 21, 27, 28, 30
insertion depth	23, 24, 31, 53, 54
orofacial	9, 11, 16, 19, 42
position	11, 12, 16, 17, 21, 23, 27, 28, 42
shoulder diameter	13–15, 19, 20, 28
specific indications	09, 10
Bone Level Implant	12, 14, 18, 20, 26, 31, 32, 35, 37, 40, 42, 49, 53, 61
Standard Implant	3, 4, 17, 31, 34, 36
Standard Plus Implant	37, 40, 53, 61
Standard Plus Short Implant	3, 4, 5, 9, 10, 17, 26, 32, 34, 37, 52
Standard Plus Narrow Neck CrossFit Implant	4–6, 8, 9, 17, 19, 20, 21, 26, 61
Tapered Effect Implant	3–5, 8, 20, 31, 32, 34, 37, 53, 61

Roxolid®	4, 9, 10
SLActive®	9, 10, 17, 27, 31, 38, 52
Titanium	9
indication	8–10, 16, 46
lingual/palatinal wall	18
mesial/distal point	18, 25
minimum guidelines	11, 26
Morse taper connection	5, 6
Pilot Drill	25, 26–28, 30, 53
preparation	
implant bed	26, 34–36, 41, 44, 45, 54, 56
basic	26, 27, 30, 35
fine	28, 29, 34–36
preparation depth	17, 28, 29, 53
profile drilling	26, 31, 34, 37
Ratchet	33, 36, 40, 41, 43, 44, 53, 56, 57
reconstruction	
prosthetic	10, 14, 19
risk of infection	
minimize	54
SCS Screwdriver	46–48, 57–59
single-patient drill	30, 54, 60
SLA® surface	9, 10, 17, 27, 31, 38, 52
Straumann® Dental Implant System	3, 56, 58, 59, 61
Drill Stop	54
Surgical Cassette	3, 26, 53, 55, 61
superstructure	11
surfaces	4, 9, 17, 25, 31, 52
synOcta® connection	4–6, 61
tapping	26, 31–35, 56–59
transfer part	18, 42, 44, 57
Twist Drill PRO	29, 30, 53, 60
wax-up/set-up	11
X-ray	
distortion	23, 24
X-ray Templates	21, 23
X-ray Reference Sphere	21, 23
X-ray Reference Sphere diameter	24

Straumann North American Headquarters

ifu.straumann.com

Straumann USA, LLC

60 Minuteman Road

Andover, MA 01810

Phone 800/448 8168 (US) • 800/363 4024 (CA)

Fax 978/747 2490

www.straumann.us • www.straumann.ca