



Loading Jobs/Parts to Netfabb

Translation of the Original Instructions

EN

About the Instructions

These instructions describe the procedure of how to load a job to Netfabb.

The personnel must read and understand this manual before beginning any work. Basic requirement for safe working is the compliance with all safety and handling instructions given in this manual.

Any passing on as well as reproduction of this document, utilization and communication of its contents are strictly prohibited unless expressly permitted. Violation obligates compensation. The original instructions were written in German.

Technical Service

Please contact your distributor or Rapid Shape.

1 General

There are several ways to upload a job to Netfabb. Depending on the file format and the amount of files you are using, please follow the instructions given.


2 Procedures

.stl File

Single File

Option 1 "Open file...":

1. Click on the button "Open file...".
2. Select the file you want to print and double-click on the file or click on the button "Open". The file will be uploaded to Netfabb.

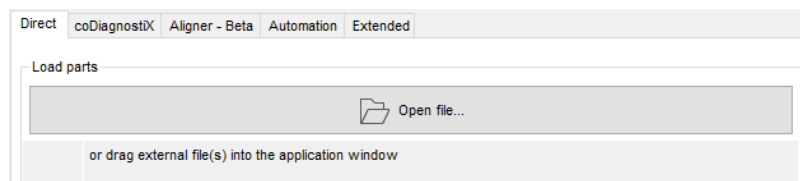


Option 2 "Drag & Drop":

1. Drag the external file and drop into the application window of Netfabb.

.stl Files

Multiple Files

1. Click on "File" in the upper left corner of Netfabb.
2. Click on "Add Part" in order to upload more files to Netfabb.

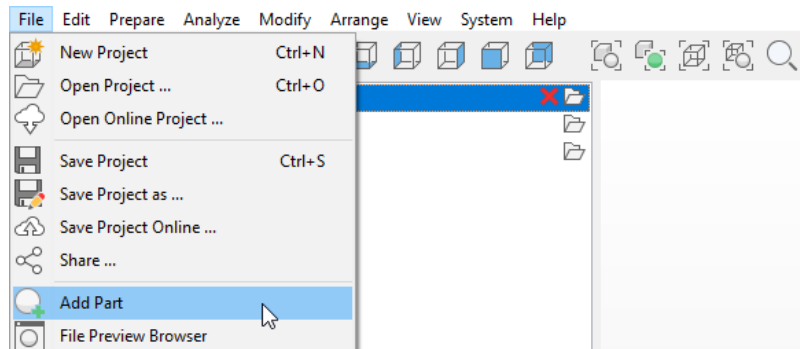

.fabbproject

1. Click on "File" in the upper left corner of Netfabb.
2. Click on "Open Project" in order to select and upload a .fabbproject.

