It's as Simple as That!


Install Your 3D Printing System on Your Network.

Dear Customer,

you have received our products with fantastic network and online features. In order to use all functions to the fullest, certain settings must be defined in your network. Either you or your IT expert can make sure that these settings are correct. This document lists all necessary information to do so. Because of special settings in some customer network systems, many details will follow. You will most likely not need all the information given as most IT systems are already setup correctly.

1. Functional Overview

Our products communicate with each other to ensure their functionality and to make faster, more convenient work processes. The following overview explains all connections between the products and what is behind them.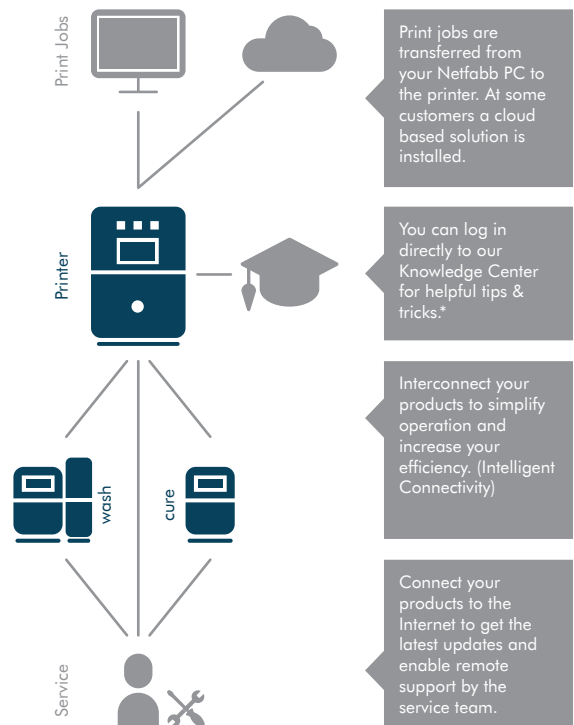


*for printers of type x30/x40 only

Please Note:

- All connections can be either wire-based (LAN) or wireless (WLAN).
- All products must be on the same network to communicate with each other.
- All products are set to DHCP at delivery. However, fixed IP addresses can also be assigned. We recommend to set DHCP and fixed IP addresses in the DHCP server (or router).
- For details on setting up and using the individual functions, please refer to the instructions for use of the respective product.
- For updates the file size for downloads should not be limited. If necessary, allow at least 200 MB. (e.g. for major function updates)

2. Overview of all IP Addresses & Ports (for IT Experts)

The following overview should help you obtain a better understanding of the product, their IP addresses and required tools. In some networks certain ports are blocked. Make sure that at least the listed ports are enabled. In order for the IT experts to know for which IP addresses they should enable the ports for, enter the IP addresses in the overview and hand this sheet over to them.

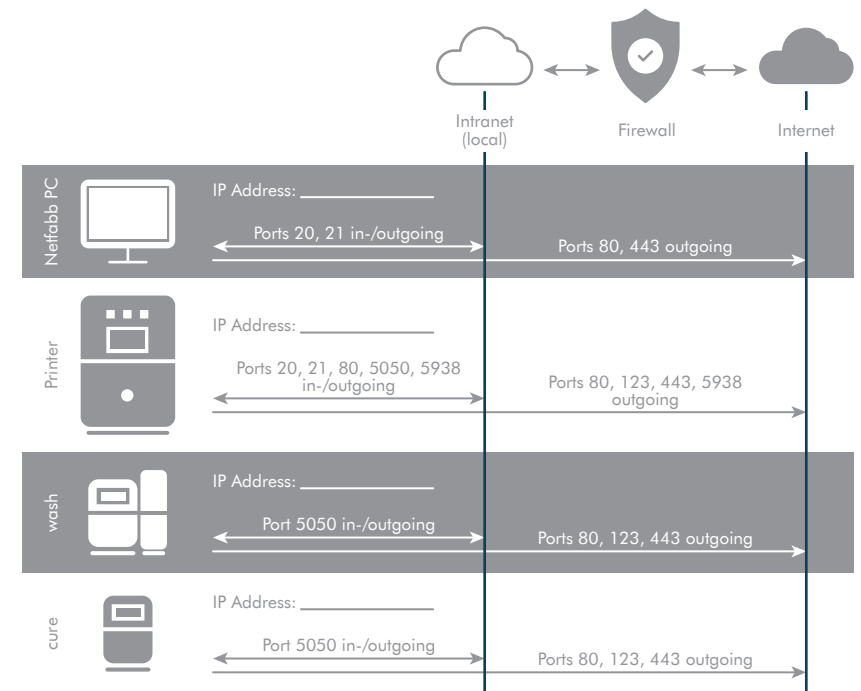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