



.8.2 Make a Pre-Preparation Scan

➡ **Step 1:** In the order form, press the **Pre-Preparation** button in the lower right corner of the window. You can do this before or after marking the tooth to be restored and selecting the indication type.

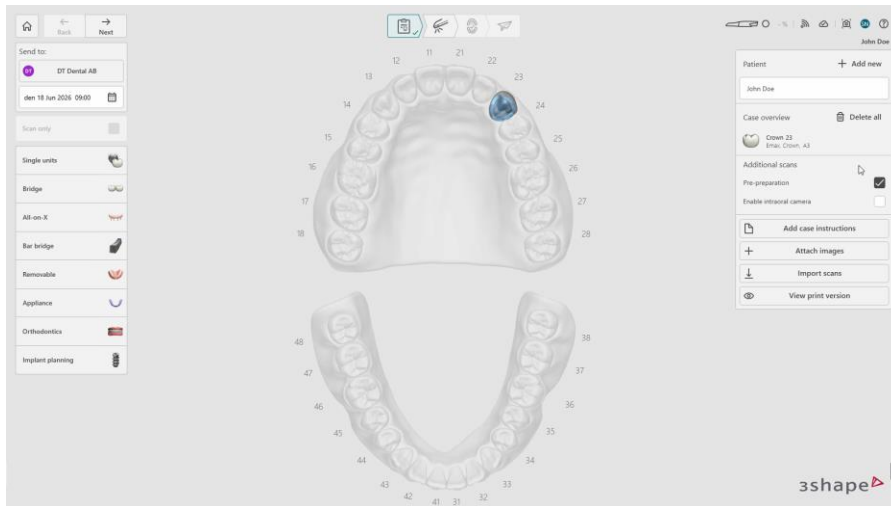


➡ **Step 2:** Proceed to the **Scan** page.


➡ **Step 3:** Make a pre-preparation scan.

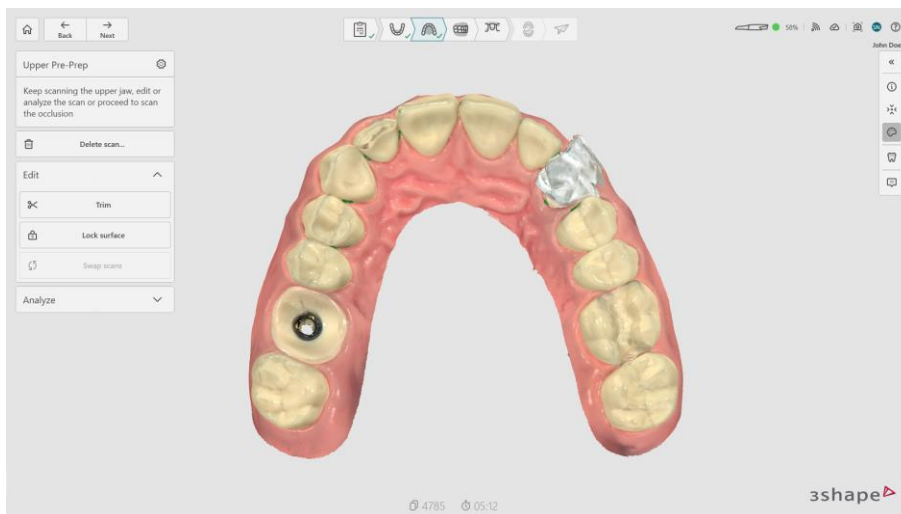

➡ **Step 4:** Proceed to the **Occlusion** page. Analyze the occlusion, edit the bite scan if needed, and go to the upper jaw (prep) page.

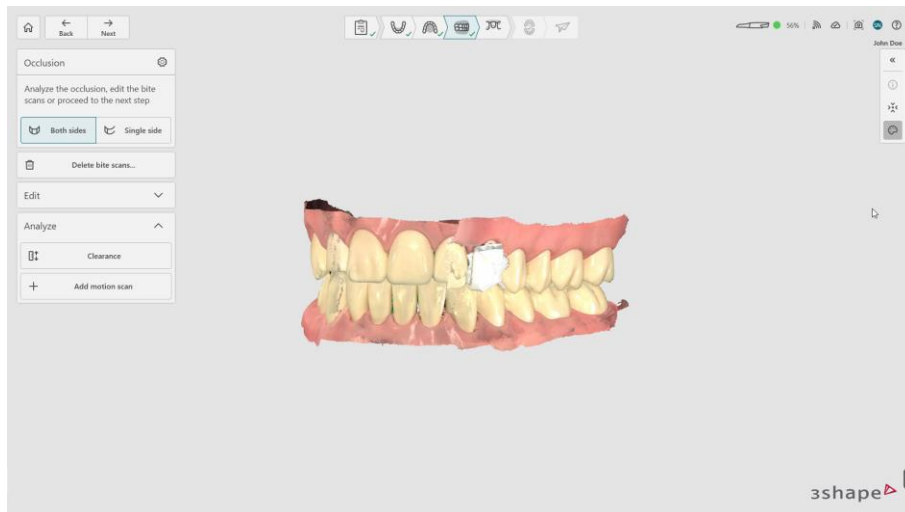

- ➡ **Step 5:** Mark the center/top of the tooth using the **Mark tooth** tool and press done.
- ➡ **Step 6:** The marked area is now auto-deleted, and the rest is locked to avoid unwanted changes to the scan.

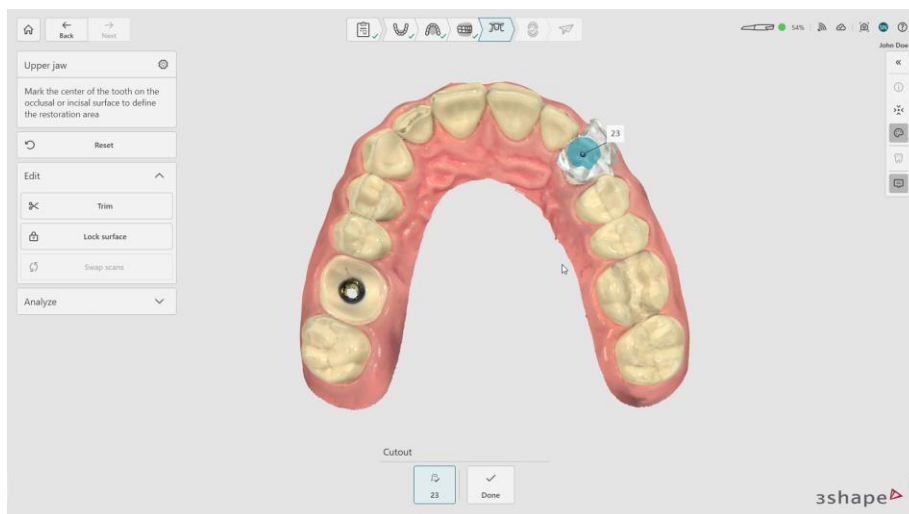

- ➡ **Step 7:** Finish your preparation and begin scanning until the deleted area is filled out again.

