

Smart Product Descriptions

Healing Caps and Healing Abutments

A versatile portfolio of Straumann® Healing Caps and Healing Abutments is available for all Straumann implants enabling soft tissue sculpturing during transmucosal healing.

In this classic one-stage surgical procedure, the implant is not covered with soft tissue during the healing phase, but the soft tissue is sutured around the Healing Cap or Healing Abutment. This provides a less invasive and time-saving treatment on your patients by avoiding a second surgical intervention.

Straumann® Healing Caps and Healing Abutments are recommended for intermediate use. After the soft tissue healing phase, replace them with the appropriate final restoration.

Implant type	Implant platform	Art.-No.		Article	Dimensions	Material	Recommended for
SP	NNC	048.071S		Sterile NNC Healing Cap reduced height	Height 3 mm	Titanium	SmartArch
		048.074S		Sterile NNC Healing Cap regular height	Height 4.5 mm		
	RN	048.034S		Sterile RN Healing Cap	Height 3 mm	Titanium	SmartOne, SmartMulti, SmartArch
		048.037S		Sterile RN Healing Cap	Height 4.5 mm		
	WN	048.039S		Sterile WN Healing Cap	Height 3 mm	Titanium	SmartOne, SmartMulti
		048.053S		Sterile WN Healing cap	Height 4.5 mm		
BLT	NC	024.2224S		Sterile NC Healing Abutment conical	Height 3.5 mm, Ø 3.6 mm	Titanium	SmartOne, SmartMulti, SmartArch
		024.2226S		Sterile NC Healing Abutment conical	Height 5 mm, Ø 3.6 mm		
	RC	024.4224S		Sterile RC Healing Abutment conical	Height 4 mm, Ø 5 mm	Titanium	SmartOne, SmartMulti, SmartArch
		024.4244S		Sterile RC Healing Abutment conical	Height 4 mm, Ø 6.5 mm		

Smart Product Descriptions

DISCLAIMER

Straumann® Smart is a blended training and education program focused on the education of general dentists who want to become surgically active in the field of dental implantology. The program is limited to information pertaining to straightforward implant cases and focuses on a reduced portfolio of products that are suitable for the treatment of such cases.

All clinical Straumann® Smart content – such as texts, medical record forms, pictures and videos – was created in collaboration with Prof. Dr. Christoph Hämmerle, Prof. Dr. Ronald Jung, Dr. Francine Brandenburg-Lustenberger and Dr. Alain Fontolliet from the University of Zürich, Clinic for Fixed and Removable Prosthodontics and Dental Material Science, Switzerland.

Straumann does not give any guarantee that Straumann® Smart provides sufficient knowledge or instruction for the dental professional to become surgically active in the field of implantology. It is the dental professional's sole responsibility to ensure that he/she has the appropriate knowledge and instruction before placing dental implants.

Straumann® Smart does not replace a careful and thorough analysis of each individual patient by a dental professional. Further, it does not imply any guarantee or warranty with regard to completeness of the information provided to the patient. It does not replace the dental professional's duty to inform the patient about the treatment, the products and the risks involved and to receive the patient's informed consent. The dental professional is solely responsible for determining whether or not a treatment or product is suitable for a particular patient and circumstances. Knowledge of dental implantology and instruction in the handling of the relevant products is always necessary and the sole responsibility of the dental professional. The dental professional must always comply with the individual product's Instructions For Use as well as all laws and regulations.

STRAUMANN DISCLAIMS, TO THE EXTENT POSSIBLE BY LAW, ANY LIABILITY, EXPRESS OR IMPLIED, AND BEARS NO RESPONSIBILITY FOR ANY DIRECT, INDIRECT, PUNITIVE, CONSEQUENTIAL OR OTHER DAMAGES, ARISING OUT OF OR IN CONNECTION WITH ANY INFORMATION PROVIDED TO PATIENTS, ERRORS IN PROFESSIONAL JUDGMENT, IN PRODUCT CHOICES OR PRACTICE IN THE USE OR INSTALLATION OF STRAUMANN PRODUCTS.

All clinical content as well as clinical and radiographic images are provided by courtesy of Prof. Dr. Christoph Hämmerle, Prof. Dr. Ronald Jung, Dr. Francine Brandenburg-Lustenberger and Dr. Alain Fontolliet from the University of Zürich, Clinic for Fixed and Removable Prosthodontics and Dental Material Science, Switzerland.

International Headquarters

Institut Straumann AG

Peter Merian-Weg 12

CH-4002 Basel, Switzerland

Phone +41 (0)61 965 11 11

Fax +41 (0)61 965 11 01

www.straumann.com

© Institut Straumann AG, 2016. All rights reserved.

Straumann® and/or other trademarks and logos from Straumann® mentioned herein are the trademarks or registered trademarks of Straumann Holding AG and/or its affiliates.