


Smart Product Descriptions

NC/RC Impression Post for closed tray

Use these Impression Posts for closed-tray impressions on Straumann® Bone Level Tapered Narrow CrossFit® or Regular CrossFit® Implants.

The benefits of closed-tray impressions are:

- No additional preparation (i.e. window, perforations) of the impression tray is required.
- The Impression Cap can be easily “snapped” into place and can be used in most cases.

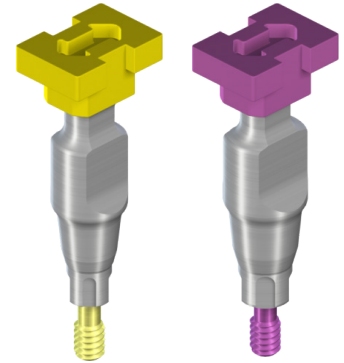

NC/RC Impression Posts for closed tray are color-coded:


- Yellow for impressions on NC Ø 3.3 mm prosthetic platforms
- Magenta for impressions on RC Ø 4.1/4.8 mm prosthetic platforms

Important


- The NC/RC Impression Posts for closed tray are supplied non-sterile and require disinfection before use.
- They are intended for single use only to ensure optimal fit and precise impression-taking for each patient.
- The Impression Posts have to be protected against aspiration when being used intraorally (e.g., use of a throat pack or thread is recommended).
- A spare cap is provided with each package in case there is a need to retake the impression immediately. The directional arrow on the cap is intended to point in a buccal-lingual/palatal direction.

Smart Product Descriptions

For Straumann® Bone Level Tapered Implants with Narrow CrossFit® connection

Art.-No.		Article	Dimensions	Material
025.2201		NC Impression Post for closed tray, with 1 guide screw and 2 caps	Length 12.3 mm	TAN/ POM

For Straumann® Bone Level Tapered Implants with Regular CrossFit® connection

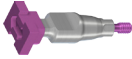Art.-No.		Article	Dimensions	Material
025.4201		RC Impression Post for closed tray, with 1 guide screw and 2 caps	Length 12.3 mm	TAN/ POM

TAN = Titanium-aluminum-niobium alloy

POM = Polyoxymethylene

Smart Product Descriptions

DISCLAIMER

Straumann® Smart is a blended training and education program focused on the education of general dentists who want to become surgically active in the field of dental implantology. The program is limited to information pertaining to straightforward implant cases and focuses on a reduced portfolio of products that are suitable for the treatment of such cases.

All clinical Straumann® Smart content – such as texts, medical record forms, pictures and videos – was created in collaboration with Prof. Dr. Christoph Hämmerle, Prof. Dr. Ronald Jung, Dr. Francine Brandenburg-Lustenberger and Dr. Alain Fontollet from the University of Zürich, Clinic for Fixed and Removable Prosthodontics and Dental Material Science, Switzerland.

Straumann does not give any guarantee that Straumann® Smart provides sufficient knowledge or instruction for the dental professional to become surgically active in the field of implantology. It is the dental professional's sole responsibility to ensure that he/she has the appropriate knowledge and instruction before placing dental implants.

Straumann® Smart does not replace a careful and thorough analysis of each individual patient by a dental professional. Further, it does not imply any guarantee or warranty with regard to completeness of the information provided to the patient. It does not replace the dental professional's duty to inform the patient about the treatment, the products and the risks involved and to receive the patient's informed consent. The dental professional is solely responsible for determining whether or not a treatment or product is suitable for a particular patient and circumstances. Knowledge of dental implantology and instruction in the handling of the relevant products is always necessary and the sole responsibility of the dental professional. The dental professional must always comply with the individual product's Instructions For Use as well as all laws and regulations.

STRAUMANN DISCLAIMS, TO THE EXTENT POSSIBLE BY LAW, ANY LIABILITY, EXPRESS OR IMPLIED, AND BEARS NO RESPONSIBILITY FOR ANY DIRECT, INDIRECT, PUNITIVE, CONSEQUENTIAL OR OTHER DAMAGES, ARISING OUT OF OR IN CONNECTION WITH ANY INFORMATION PROVIDED TO PATIENTS, ERRORS IN PROFESSIONAL JUDGMENT, IN PRODUCT CHOICES OR PRACTICE IN THE USE OR INSTALLATION OF STRAUMANN PRODUCTS.

All clinical content as well as clinical and radiographic images are provided by courtesy of Prof. Dr. Christoph Hämmerle, Prof. Dr. Ronald Jung, Dr. Francine Brandenburg-Lustenberger and Dr. Alain Fontollet from the University of Zürich, Clinic for Fixed and Removable Prosthodontics and Dental Material Science, Switzerland.

International Headquarters

Institut Straumann AG

Peter Merian-Weg 12

CH-4002 Basel, Switzerland

Phone +41 (0)61 965 11 11

Fax +41 (0)61 965 11 01

www.straumann.com

© Institut Straumann AG, 2016. All rights reserved.

Straumann® and/or other trademarks and logos from Straumann® mentioned herein are the trademarks or registered trademarks of Straumann Holding AG and/or its affiliates.